READ THIS MANUAL COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT

Keep this manual in a safe location for future reference

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CONTENT

| | |
|--|---|
| 1. IMPORTANT-SAFETY-INSTRUCTIONS | |
| - DO NOT DESTROY..... | 2 |
| 2. Package Contents | 4 |
| 3. Parts and Controls..... | 5 |
| 4. Intended Use | 5 |
| 5. To Use..... | 5 |
| 6. Care, Maintenance and Disposal..... | 6 |
| 7. Technical Specifications | 7 |
| 8. Troubleshooting Guide..... | 7 |
| 9. FCC Compliance Information | 8 |
| 10. Warranty | 8 |

SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE

If the instruction manual is damaged or if you no longer have the instruction manual in your possession, please contact customer service. See warranty for service contact.

Dear Valued Customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for application in the areas of heating, weight management, blood pressure, body temperature, pulse, gentle therapy, massage, and beauty.

Register your product today to receive further benefits:

- added protection for your purchase
- health and wellness tips
- new product information
- continual outstanding customer service and support.

Visit www.registerbeurer.com or contact customer service at 1-800-536-0366.

Please read these instructions for use carefully and keep them for later use.



Make them accessible to other users and observe the information they contain.

With kind regards,

Your Beurer team

1. IMPORTANT-SAFETY- INSTRUCTIONS - DO NOT DESTROY

READ THIS ENTIRE MANUAL, THE SAFETY SECTION AND ALL INSTRUCTIONS AND WARNINGS COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT. FOLLOW ALL SAFETY INSTRUCTIONS AND WARNINGS TO AVOID HAZARDOUS SITUATIONS AND TO MAKE CORRECT USE OF THIS PRODUCT.
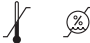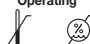
| | |
|---|--|
|  | This is the safety alert symbol. It alerts you to potential personal injury hazards. Obey all safety messages that follow this symbol to avoid possible injury or death. |
|  WARNING | WARNING indicates a hazardous situation which, if not avoided, could result in death or serious injury. |
|  CAUTION | CAUTION indicates a hazardous situation which, if not avoided, may result in minor or moderate injury. |
|  NOTICE | NOTICE addresses practices not related to personal injury, such as product and/or property damage. |
| Storage  | Permissible storage and transport temperature and humidity |
| Operating  | Permissible operating temperature and humidity |
|  | Observe the instructions for use |

 **WARNING – To reduce the risk of burns, electric shock and fire, this product must be used in accordance with the following instructions:**

- Read all instructions before using the product.
- Do not use on infants.
- Keep children away from packaging materials (risk of suffocation).
- Never leave heating pad unattended.
- Do not use pins or other metallic means to fasten this pad in place.
- Never pull this pad by the power cord. Do not use the cord as a handle. Unplug when not in use.
- Do not use a heating pad when symptoms of appendicitis are present. Consult your physician.
- Inspect this product before using it in order to make sure that there are no broken parts, missing parts and/or damaged areas. Do not use heating pad if there are broken parts, missing parts or damaged areas.
- This appliance has a polarized plug (one blade is wider than the other). As a safety feature, this plug will fit in a polarized outlet only one way. If the plug does not fit fully in the outlet, reverse the plug. If it still does not fit, contact a qualified electrician. Do not attempt to defeat this safety feature.
- Do not use an electrical outlet that has become loose or does not engage the power plug tightly.
- Do not immerse in water or use chemicals or solvents during cleaning, except as directed in the owner's manual.
- Do not remove product labels that contain warnings or safety instructions.
- Consult with a physician for use with controlled substances.
- Do not tuck in the wired area of this heating pad.
- Do not route electrical cords between the mattress and boxspring. Do not allow the cords to be pinched.
- Keep the controller away from drafty areas that may become damp or wet such as an open window.
- Do not use this heating pad if you or the product is wet.
- Be sure that this heating pad is used on an alternating current supply circuit of the proper voltage.
- Turn off and unplug when not in use.
- Indoor use only.

- With the heating pad switched on, never place any sharp object or any heating source such as a hot-water bottle, heating pad or similar items on it.
- The electronic components inside the control unit will heat up whilst operating the heating pad. Therefore never cover the control unit or place it onto the heating pad during operation.
- The electronic and magnetic fields emitted by this heating pad may interfere with the function of a pacemaker. However, they are still well below the limits for electrical field strength (5000 V/m max), magnetic field strength (80 A/m max), and magnetic flux density (0.1 millitesla max). Please consult your doctor and the manufacturer of your pacemaker before using this heating pad.
- Household plug use only.

CAUTION – To reduce the risk of personal injury or product/property damage:

- Do not use while sleeping.
- This pad should not be used on or by persons who are unconscious, persons with poor blood circulation, or persons with diabetes, unless they are carefully monitored.
- Do not use on areas of insensitive skin.
- Burns can occur regardless of control setting. Check skin under pad frequently.
- Do not use pad with liniment, salve or ointment preparations that contain heat-producing ingredients. This can result in skin burns.
- Do not use with alcohol.
- Do not tuck the heating pad or supply cord in a sofa or other furniture. All portions of the heating pad or controller should be visible at all times.
- Do not trap or pinch the cord.
- Protect the heating pad from knocks, damp, dust, chemicals, marked temperature fluctuations and nearby sources of heat (ovens, heaters).
- Do not use this heating pad on animals. Keep all pets away.

NOTICE

- Do not crush pad. Avoid sharp folds.
- This heating pad is intended for use on top of the body. Do not sit or lie on top of the heating pad. Never place pad between yourself and a chair, sofa, bed or pillow.
- Loop cord loosely when storing. Tight wrapping may damage cord and internal parts.
- Do not use when folded or bunched up.
- The heating pad is not suitable for use in hospitals or for any other commercial use.
- Not for Hotel, Motel or other institutional use.
- Not for medical use in hospitals.
- Only use the heating pad together with the control unit specified on the label. Otherwise the heating pad could be damaged.


2. Package Contents

1x Heating Pad UHP 24

1x Controller with Power Cord

1x Instruction Manual

3. Parts and Controls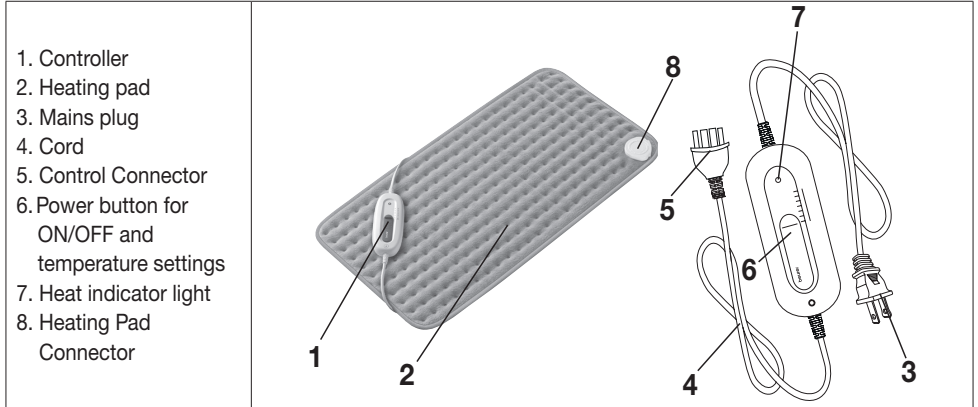

4. Intended Use

This heating pad is intended for applying heat to the human body for warmth and comfort. It is not suitable for use in hospitals or for any other commercial use. In particular, it should not be used for babies or infants, persons who are insensitive to heat, helpless persons, or animals. Any use not described above, or modifications of the heating pad are not permissible and may lead to injury and /or damage to the heating pad. The manufacturer assumes no liability for damage due to improper use.

Safety System

The heating pad is equipped with a safety system. The electronic sensor technology prevents the entire surface of the heating pad from overheating by automatically switching off in the event of a fault. If the safety system switches the heating pad off automatically due to a fault, then the heating indicator light on the controller is no longer illuminated, even if the heating pad is switched on.

NOTICE


- Please note that, for safety reasons, the heating pad must not be used after a fault and must be returned to the service address indicated.
- Never connect the defective heating pad to another controller of the same type. This would also result in a permanent shutdown by the safety system in the controller.

5. To Use

NOTICE

- The heating pad may smell of plastic the first time you use it, but this will disappear after a short period of time.

To begin, first connect the control connector to the heating pad connector.

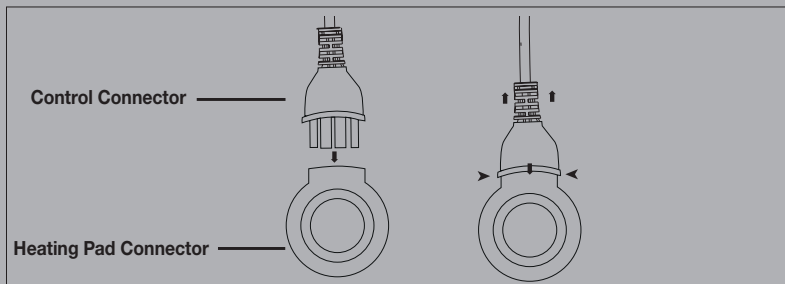The control connector must be connected to the heating pad connector before plugging the unit into an electrical outlet. After connecting the heating pad connector to the control connector, plug the unit into a 120V AC electrical outlet.

Switching On

Set the controller to 1, 2, 3, 4, 5 or 6 to switch on the heating pad.

NOTICE

- The heat indicator light illuminates after the heating pad is switched on.
- This heating pad features rapid heating, which allows it to heat up within the first 10 minutes of operation.

Switching Off

Set the controller to 0 to switch off the heating pad.

NOTICE

- The heat indicator light will switch off.

Selecting Temperature

Just before using the heating pad, set it to the highest temperature setting (setting 6). This will cause the heating pad to warm up quickly. When using the heating pad for several hours, set it to the lowest temperature setting (setting 1).

Automatic shut-off

The heating pad switches itself off automatically after about 120 minutes. The heat indicator light then starts to flash. Then select the setting 0 and after about 5 seconds, reset the heating pad to the desired temperature setting to switch it on again.

If the heating pad is no longer used after the timer expires:

- Switch off the heating pad (setting 0),
- Unplug from electrical outlet,
- Disconnect the controller from the heating pad.

6. Care, Maintenance and Disposal

Washing Instructions

DO NOT DRY CLEAN! DO NOT USE BLEACH! DO NOT USE WRINGER! DO NOT IRON YOUR HEATING PAD!

If the heating pad is slightly soiled, use a dampen cloth and some mild detergent, if necessary. Use a dry, lint-free cloth for controller cleaning and care.

WARNING

- Disconnect the mains plug. Do NOT pull by the cord.
- Do not reconnect the controller to the heating pad until the control connector and the heating pad are completely dry. Otherwise there is a risk of electric shock.
- Never switch the heating pad on to help it dry. Otherwise there is a risk of electric shock

NOTICE

- Disengage pad/controller connectors.
- Never immerse the controller in water or other liquids. It may otherwise be damaged.
- Do NOT use chemical cleaning or scouring agents to clean the heating pad and controller. The heating pad and controller may otherwise be damaged.
- Machine wash pad in warm water on gentle cycle; tumble dry on medium heat. Remove promptly from the dryer. Or, hand wash in warm water and line dry.
- Do NOT plug in the heating pad until it is dry.

- Reshape the still damp heating pad to the original size immediately after washing, and dry flat, spread on a clothes drying rack.
- Do not hang the heating pad up to dry with pegs or similar items. The heating pad may otherwise be damaged.

Storage

NOTICE

- Let the heating pad cool before putting it away. The heating pad may otherwise be damaged.
- To prevent the heating pad from becoming sharply creased, do not store other objects on top of the heating pad. The heating pad may otherwise be damaged.
- If you do not intend to use the heating pad for any length of time, store it in the original packaging in a dry environment and without weighing it down.
- Disconnect the mains connection and the controller from the heating pad to do so.

Disposal

Observe the local regulations for material disposal. Dispose of the device in accordance with local regulations. If you have any questions, please contact the local authorities responsible for waste disposal.

7. Technical Specifications

| | |
|-----------------------|--|
| Model: | UHP 24 |
| Type: | U-SP16R |
| Heating Pad Size: | 12 x 15 in (30.5 x 38 cm) |
| Power supply: | 120 V / 60 Hz |
| Wattage: | 50 W |
| Pad Material: | 100% polyester |
| Timer: | 120 minutes |
| Temperature levels: | 6 levels Level 0: OFF Level 1: minimum heat Level 2-5: medium heat Level 6: maximum heat |
| Operating Conditions: | 32°F to 95°F (0°C - 35 °C) < 80% humidity |
| Storage Conditions: | 23°F to 104°F (-5°C to 40°C) < 80% humidity |

8. Troubleshooting Guide

| Problem | Cause | Solution |
|---|---|--|
| The heat indicator light is not illuminated while <ul style="list-style-type: none"> • The controller is properly connected to the heating pad. • The mains plug is connected to the working electrical outlet. • The sliding switch is set to 1,2,3,4,5 or 6. | The safety system has switched the heating pad off permanently. | Send the heating pad and the controller for servicing. |

9. FCC Compliance Information

Heating Pad UHP 24

Responsible Party – U.S. Contact Information:

Beurer North America LP

1 Oakwood Boulevard, Suite 255

Hollywood, FL 33020

United States

1-800-536-0366

info@beurer.com

FCC Compliance Statement

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

1. This device may not cause harmful interference, and
2. This device must accept any interference received, including interference that may cause undesired operation.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

10. Warranty

Limited Lifetime Warranty For Original Purchaser

Your Beurer Heating Pad, model UHP 24, is warranted to be free from defects in materials and workmanship for the life of the product under normal conditions of intended use and service. This warranty extends only to the original retail purchaser and does not extend to retailers or subsequent owners.

We will, at our option, repair or replace the Beurer Heating Pad, model UHP 24, without additional charge, for any part or parts covered by these written warranties. No refunds will be given. Repair or replacement is our only responsibility and your only remedy under this written warranty. If replacement parts for defective materials are not available, Beurer reserves the right to make product substitutions in lieu of repair or replacement.

For warranty service contact our customer service department at 1-800-536-0366 or at info@beurer.com to provide a description of the problem. If the problem is deemed to be within the scope of the limited lifetime warranty, you will be asked to mail the product at your costs in its original package with proof of purchase, your name, address and phone number. If the problem is not deemed to be within the scope of the limited lifetime warranty, we will provide a quotation for repair respectively replacement and return shipping fee.

This warranty does not cover damage caused by misuse or abuse; accident; the attachment of unauthorized accessory; alteration to the product; improper installation; misapplication; lack of reasonable care with respect to the product; unauthorized repairs or modifications; improper use of electrical/power supply; old worn

batteries; normal wear; loss of power; dropped product; malfunction or damage of an operating part as a result of failure to comply with instructions for use or to provide manufacturer's recommended maintenance; transit damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Beurer. This warranty is void if the product is ever used in a commercial or business environment. The maximum liability of Beurer under this warranty is limited to the purchase price actually paid by the customer for the product covered by the warranty, as confirmed by proof of purchase, regardless of the amount of any other direct or indirect damage suffered by the customer.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adaptation to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. ANY IMPLIED WARRANTIES, OBLIGATIONS, OR LIABILITIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTY OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED IN DURATION TO THE DURATION OF THIS APPLICABLE WRITTEN WARRANTY. Some states do not allow limitations on how long an implied warranty lasts, so the above limitations may not apply to you.

IN NO EVENT SHALL BEURER BE LIABLE FOR ANY SPECIAL, INCIDENTAL, INDIRECT OR CONSEQUENTIAL DAMAGES FOR BREACH OF THIS OR ANY OTHER WARRANTY, EXPRESS, IMPLIED OR ANY OTHER THEORY OF LIABILITY, WHATSOEVER. Some states do not allow the exclusion or limitation of special, incidental, or consequential damages, so the above limitation may not apply to you.

Beurer does not authorize anyone, including, but not limited to, retailers, the subsequent consumer purchaser of the product from a retailer or remote purchaser, to obligate Beurer in any way beyond the terms set forth herein.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate in connection with any products or parts thereof which are repaired, replaced, altered, or modified, without the prior explicitly written consent of Beurer.

This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state.

For more information regarding our product line in the USA, please visit: www.beurer.com

Distributed by:
Beurer North America LP
1 Oakwood Boulevard, Suite 255
Hollywood, FL 33020, USA
www.beurer.com

For Canada only:
Manufacturer:
Beurer GmbH
Söflinger Str. 218
89077 Ulm, Germany

Questions or comments? Call our US-based customer service toll free at 1-800-536-0366.

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