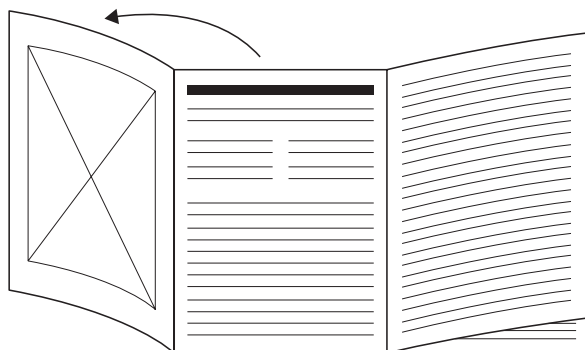
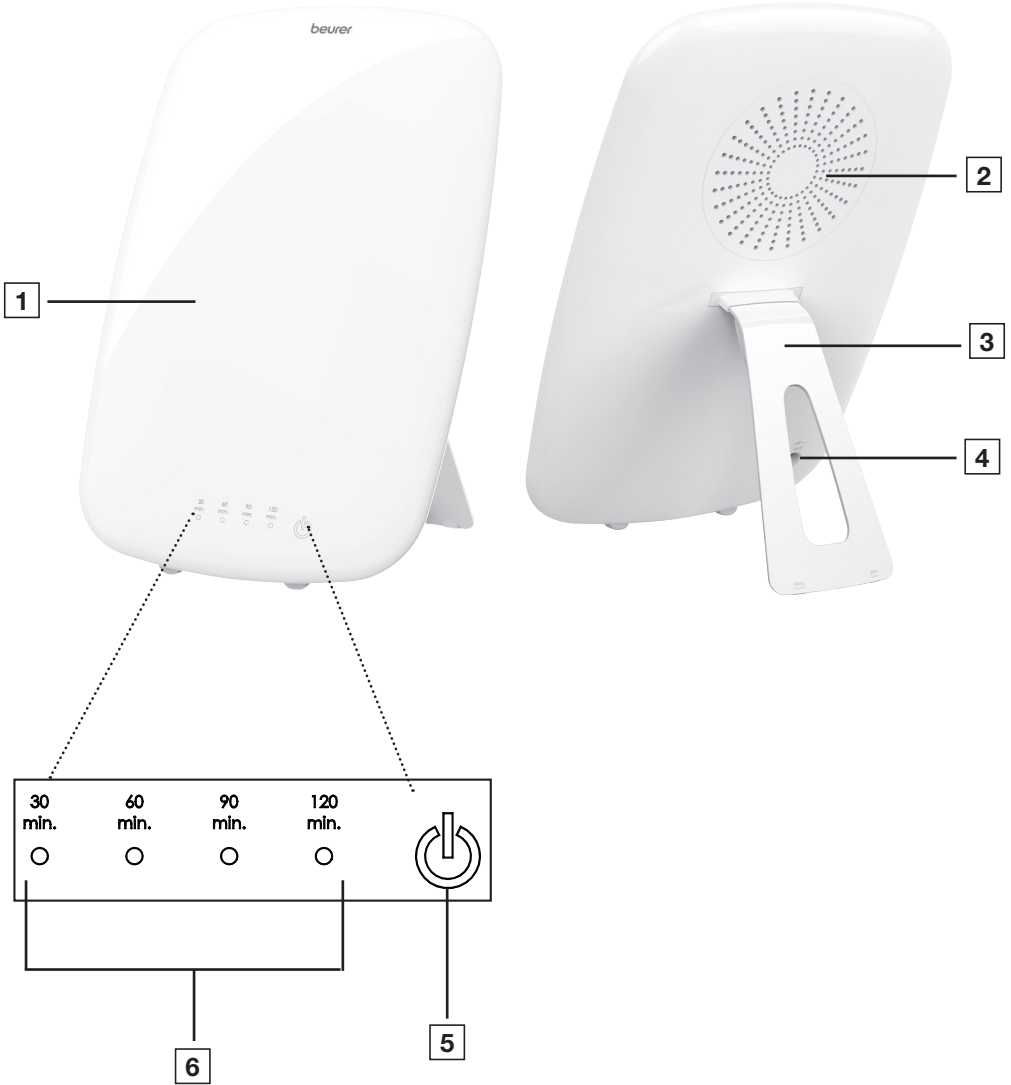
DE Tageslichtlampe Gebrauchsanweisung	4	PL Lampa o świetle dziennym Instrukcja obsługi	79
EN Daylight therapy lamp Instructions for use.....	15	NL Daglichtlamp Gebruiksaanwijzing	90
FR Lampe de luminothérapie Mode d'emploi.....	25	DA Lysterapilampe Betjeningsvejledning.....	100
ES Lámpara de luz diurna Instrucciones de uso.....	36	SV Dagsljuslampa Bruksanvisning.....	110
IT Lampada di luce naturale del giorno Istruzioni per l'uso.....	47	NO Dagslys Lampe Bruksanvisning.....	120
TR Gün ışığı lambası Kullanım kılavuzu.....	58	FI Kirkasvalolamppu Käyttöohje.....	130
RU Прибор дневного света Инструкция по применению	68		

- DE** Klappen Sie vor dem Lesen der Gebrauchsanweisung die Seite 3 aus.
- EN** Unfold page 3 before reading the instructions for use.
- FR** Dépliez la page 3 avant de lire le mode d'emploi.
- ES** Despliegue la página 3 antes de leer las instrucciones de uso.
- IT** Prima di leggere le istruzioni per l'uso aprire la pagina 3.
- TR** Kullanım kılavuzunu okumadan önce 3. sayfayı dışa doğru açın.
- RU** Перед ознакомлением с инструкцией по применению разложите страницу 3.
- PL** Przed przeczytaniem instrukcji obsługi otworzyć stronę 3.
- NL** Vouw pagina 3 uit voordat u de gebruiksaanwijzing gaat lezen.
- DA** Fold side 3 ud, før du læser betjeningsvejledningen.
- SV** Vik ut sidan 3 innan du läser bruksanvisningen.
- NO** Åpne side 3 før du leser bruksanvisningen.
- FI** Avaa sivu 3 ennen käyttöohjeen lukemista.

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Read these instructions for use carefully. Observe the warnings and safety notes. Keep these instructions for use for future reference. Make the instructions for use accessible to other users. If the device is passed on, provide the instructions for use to the next user as well.

Contents

1. Signs and symbols	16	8. Cleaning and maintenance.....	21
2. Intended purpose	17	9. Accessories and replacement parts.....	21
3. Warnings and safety notes	17	10. What if there are problems?	21
4. Included in delivery	19	11. Disposal.....	22
5. Device description.....	19	12. Technical specifications	23
6. Initial use	19	13. Warranty	24
7. Usage	20		

Why use a daylight therapy lamp?

When the hours of daylight are noticeably shorter in autumn and people increasingly stay inside in winter, the effects of a lack of light may become apparent. This is often described as “winter depression”. The symptoms can present themselves in a number of ways:

- Imbalance
- Subdued mood
- Lack of energy and listlessness
- Generally feeling under the weather
- Need for more sleep
- Loss of appetite
- Difficulty concentrating

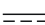
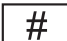
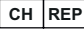
The cause of these symptoms is the fact that light – particularly sunlight – is essential for life and has a direct effect upon the human body. Sunlight indirectly controls the production of melatonin, which is only passed to the blood in darkness. This hormone shows your body that it’s time to sleep. That’s why more melatonin is produced in months with less sunshine, making it difficult to get up in the mornings because your body functions are powered down. Use the daylight therapy lamp immediately after waking up (i.e. as early as possible) to end the production of melatonin and to brighten your mood.

Lack of light also prevents the production of the happy hormone serotonin, which is said to significantly influence our well-being. The application of light thus yields quantitative changes to hormones and neurotransmitters in the brain that have an effect on our activity levels, our feelings and our well-being. To compensate for such a hormonal imbalance, daylight therapy lamps can create a suitable replacement for natural sunlight.

In the medical field, daylight therapy lamps are used to combat the effects of a lack of light. Daylight therapy lamps simulate daylight over 10,000 lux. This light can influence the human body and be used as a treatment or as a preventative measure. Normal electric light, however, is not sufficient to influence the hormonal balance. This is because in a well-lit office, the light intensity is just 500 lux, for example.

1. SIGNS AND SYMBOLS

The following symbols are used on the device, in these instructions for use, on the packaging and on the type plate for the device:

	WARNING Warning notice indicating a risk of injury or damage to health		CE labelling This product satisfies the requirements of the applicable European and national directives.
	IMPORTANT Safety note indicating possible damage to the device/accessory		Permissible storage temperature and humidity
	Product information Note on important information		Permissible operating temperature and humidity
	Observe the instructions Read the instructions before starting work and/or operating devices or machines	IP21	Protected against solid foreign objects 12.5 mm in diameter and larger, and against vertically falling drops of water
		IP24	Protection against solid objects greater than 12,5mm and against splashes of water from all directions.
	Manufacturer		Serial number
	Protection class II device The device is double-insulated and is, therefore, in protection class 2		Dispose of packaging in an environmentally friendly manner
	ON/OFF		Marking to identify the packaging material. A = Material code, B = Material number: 1-7 = Plastics, 20-22 = Paper and cardboard
	Medical device		For indoor use only
	Item number		Authorised representative in the European Community
	Date of manufacture		Importer symbol
	atmospheric pressure limitation		Unique Device Identifier (UDI) for unique product identification
	Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE		Direct current The device is suitable for use with direct current only
	Batch Code		Type
	Separate the product and packaging elements and dispose of them in accordance with local regulations.		Swiss authorised representative

2. INTENDED PURPOSE

Purpose

The Bright Light Therapy lamp is intended to compensate the effects of lack of light, particularly sunlight, and provide relief from winter seasonal affective disorders, mood disorders and circadian phase sleep disorders.

Patient population

Adult and child over 3 years old.

Intended users

The use of the device does not require a specific knowledge or professional ability. The patient is the intended operator except in case patient that required special assistance.

Indication

The device simulates daylight to provide relief from seasonal or mood disorders.

3. WARNINGS AND SAFETY NOTES

WARNING

- The daylight therapy lamp is intended for radiation on human bodies only.
- Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Services address.
- Ensure that the daylight therapy lamp is positioned on a stable base.
- The device must only be connected to the mains voltage that is specified on the type plate.
- Never submerge the device in water and do not use it in the bathroom.
- Do not use the daylight therapy lamp on animals!
- Do not use device on patient is taking a photosensitizing medication or herb.
- Do not use device on:
 - children under age 3
 - people insensitive to heat
 - people with skin lesions due to illness
 - people that has a condition that might render his or her eyes more vulnerable to phototoxicity
 - people that has a photosensitive skin condi
- Keep packaging material away from children (risk of suffocation).
- If the device is warm, do not cover it or pack and store it.
- Always unplug the mains adapter and allow the device to cool down before touching it.
- Do not touch the device with wet hands when it is plugged in and do not allow water to spray on the device. Only operate the device if it is completely dry.
- Ensure that you only insert and remove the mains adapter with dry hands and that you only press the On/Off button with dry hands.
- Keep the mains cable away from hot objects and naked flames.
- Protect the device from heavy impacts.
- Do not pull the mains adapter out of the socket using the mains cable.
- Do not use the device if it shows signs of damage or does not function properly. In such cases, contact Customer Services.
- In the event of damage to the mains connection cable for this device, the cable must be disposed of. If it is not a removable cable, the device must be disposed of.
- Disconnection from the power supply network is only guaranteed when the mains adapter is unplugged.

- Do not use the device in the presence of flammable anaesthetic gas connections with air, oxygen or nitrogen oxide.
- Do not leave the equipment unattended when it is switched on to avoid the risk of fire or burns.
- This equipment is not intended for use by children under three years. Children under three years should be supervised to ensure that they do not play with the equipment to avoid the risk of fire and burns.
- If the wall socket used to power the equipment has poor connections, the plug of the equipment becomes hot. Make sure you plug the equipment into a properly installed wall socket to avoid the risk of fire and burns.
- Do not subject the equipment to heavy shocks to avoid risk of damage to the lamp.
- No calibration and no preventive checks or maintenance need to be carried out on this device.
- You cannot repair the device. The device contains no parts that you can repair.
- Do not make any changes to the device without the manufacturer's permission.
- If the device has been changed, thorough tests and checks must be carried out to ensure the continued safety of the device for any future use.
- To avoid strangulation and entanglement, keep cable out of reach of young children.
- The device is intended for use by the patient themselves. The device and accessories must be used in accordance with these instructions for use.

General notes

WARNING

- This device must not be used by people with severe depression or with reduced physical, sensory or mental skills or a lack of experience or knowledge.
- Diabetics and people who suffer from retinal diseases must be examined by an optician before using the daylight therapy lamp.
- Please do not use the device if you suffer from an eye disease such as cataracts, glaucoma, diseases of the optic nerve or inflammation of the vitreous body.
- Please do not use in case of a recent eye surgery or a diagnosed eye condition whereby doctor has advised you to avoid bright light.
- It must not be used on people with disabilities, children under 3 years of age or people with reduced sensitivity to heat (people with skin alterations due to illness).
- Severe cases of SAD should, in any case, be closely supervised by a clinician.
- Always consult a doctor before using the daylight therapy lamp if you are taking medication such as pain relief medication, medication to reduce high blood pressure or antidepressant medication.
- If you have health concerns of any kind, consult your general practitioner!
- Remove all packaging material before using the device.
- Light sources are excluded from the warranty.
- If the device has been in storage or recently transported, keep it for at least two hours at room temperature before using it.
- Power adapter is part of the ME equipment.
- Power-on check item: please check whether light flashes, dark areas/shadows and other abnormalities occur after power-on. If there is any abnormality, please contact the after-sales hotline.
- The patient cannot undergo MRI scan while using this device.
- Please report any serious incident that has occurred in relation to the device injury or adverse event to the local competent authority and to the Manufacturer or to the European Authorised Representative (EC REP) Vigilance contact point: <https://ec.europa.eu/growth/sectors/medical-devices/contacts/>
- PRC is the abbreviation of the People's Republic of China.

Instruction for repairs

IMPORTANT

- Do not open the device. Do not attempt to repair the device yourself. This could result in serious injury. Failure to comply with this instruction will void the warranty.
- For repairs, please contact Customer Services or an authorised retailer.

4. INCLUDED IN DELIVERY

Check that the exterior of the cardboard delivery packaging is intact and make sure that all contents are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Services address.

- 1 daylight therapy lamp
- 1 stand
- 1 set of instructions for use
- 1 mains adapter

5. DEVICE DESCRIPTION

The corresponding drawings are shown on page 3.

- | | |
|-------------------------------------|----------------------------------|
| 1 Fluorescent screen | 4 Mains adapter connector |
| 2 Rear of the device housing | 5 On/Off button |
| 3 Stand | 6 Timer setting |

6. INITIAL USE

Take the device out of the plastic wrapping. Check the device for damage or faults. If you notice any damage or faults on the device, do not use it and contact Customer Services or your supplier.

Attach stand

Attach the stand **3** to the device on the rear of the device housing.

Setting up the device

Place the device on a level surface. The position should be chosen to ensure a distance of between 25 cm and 45 cm between the user and the device. The lamp is most effective at this distance.

Mains connection

- To prevent possible damage to the device, the daylight therapy lamp must only be used with the mains adapter described here.
- Insert the mains adapter into the connector provided for this purpose on the rear of the daylight therapy lamp. The mains adapter must only be connected to the mains voltage that is specified on the type plate.
- After using the daylight therapy lamp, unplug the mains adapter from the mains socket first and then disconnect it from the daylight therapy lamp.

Note

Ensure that there is a mains socket close to the set-up area. Arrange the mains cable in such a way that no-one will trip over it.

7. USAGE

1	Insert the mains adapter into the connector.												
2	Switching on the lamp Press and hold the On/Off button 5 for 2 seconds. When the device is next switched on, it will start with the saved brightness level and treatment time.												
3	LED display/timer The TL 85 Daylight therapy lamp indicates your current treatment time using 4 LEDs. The treatment time can be set at the following 4 levels. The timer function will not be active when the lamp is switched on for the first time. The lamp now runs until it is switched off again or a treatment time is set (timer = 0 min). In order to use the timer, set the desired level from 1 to 4 by pressing briefly the On/Off button 5 . Level 1 sets a treatment time of 30 minutes. The other levels are as follows: <table border="1" data-bbox="116 453 624 616"><thead><tr><th>Treatment time</th><th>Level (Number of lit LEDs)</th></tr></thead><tbody><tr><td>30 minutes</td><td>1</td></tr><tr><td>60 minutes</td><td>2</td></tr><tr><td>90 minutes</td><td>3</td></tr><tr><td>120 minutes</td><td>4</td></tr></tbody></table> The lamp switches off automatically after this treatment time and steadily becomes darker. To deactivate the timer function, briefly touch the On/Off button 5 until the LEDs go out.	Treatment time	Level (Number of lit LEDs)	30 minutes	1	60 minutes	2	90 minutes	3	120 minutes	4		
Treatment time	Level (Number of lit LEDs)												
30 minutes	1												
60 minutes	2												
90 minutes	3												
120 minutes	4												
4	Enjoying the light Position yourself as close as possible to the lamp, at a distance of between 25 cm and 45 cm. You can continue to go about your day-to-day activities during the treatment, including reading, writing, making telephone calls, etc. <ul data-bbox="127 810 1057 1058" style="list-style-type: none">• Keep looking directly into the light for a short period of time, as it is absorbed and shows an effect via the eyes/retina.• Use the daylight therapy lamp as often as you want. However, the treatment is most effective if you carry out the light therapy for at least 7 successive days according to the prescribed times.• The most effective time of day for the treatment is between 6 am and 8 pm, and we recommend that you use the device for 2 hours per day.• However, do not look directly into the light for the entire treatment time, as this may cause over-stimulation of the retina.• Start with a brief treatment time and gradually increase the time over the course of a week. <p>i Note</p> <p>You may experience pain in the eyes and headaches after the first few applications. This pain should disappear in further sessions as the nervous system becomes accustomed to the new stimuli.</p>												
5	Things to consider We recommend a distance of between 25 and 45 cm from the face to the lamp during treatment. The treatment time depends on the distance: <table border="1" data-bbox="116 1278 589 1433"><thead><tr><th>Lux</th><th>Distance</th><th>Treatment time</th></tr></thead><tbody><tr><td>10,000</td><td>approx. 25 cm</td><td>0.5 hours</td></tr><tr><td>5,000</td><td>approx. 35 cm</td><td>1 hour</td></tr><tr><td>2,500</td><td>approx. 45 cm</td><td>2 hours</td></tr></tbody></table> <p>In principle: The closer you are to the light source, the shorter the treatment time.</p>	Lux	Distance	Treatment time	10,000	approx. 25 cm	0.5 hours	5,000	approx. 35 cm	1 hour	2,500	approx. 45 cm	2 hours
Lux	Distance	Treatment time											
10,000	approx. 25 cm	0.5 hours											
5,000	approx. 35 cm	1 hour											
2,500	approx. 45 cm	2 hours											

6	Extra bright This daylight therapy lamp is particularly bright (14,000 lux at a distance of approx. 20 cm).
7	Enjoying the light over longer periods Repeat the treatment on at least 7 consecutive days during darker periods of the year, or for longer depending on your individual needs. The treatment should take place during the mornings where possible.
8	Switching off the lamp Press and hold the On/Off button  for 2 seconds. The LEDs switch off. Unplug the mains adapter from the mains socket.  IMPORTANT The lamp remains hot after use. Allow the lamp to cool down sufficiently before putting it away and/or packing it away.

8. CLEANING AND MAINTENANCE

The device should be cleaned from time to time.

IMPORTANT

- Ensure that no water gets inside the device!
The device must be switched off, disconnected from the mains and allowed to cool down each time before cleaning.
- Do not clean the device in the dishwasher. Use a slightly damp cloth to clean the device.
- Do not use any abrasive cleaning products and never submerge the device in water.
Do not touch the device with wet hands when it is plugged in and do not allow water to spray on the device. Only operate the device if it is completely dry.

Storage

If you are not going to use the device for an extended period of time, disconnect it and store it in a dry place, out of the reach of children.

Follow the storage instructions provided in the “Technical specifications” chapter.

9. ACCESSORIES AND REPLACEMENT PARTS

To purchase accessories and replacement parts, please visit www.beurer.com or contact the service address for your country (see the service address list). Accessories and replacement parts are also available from retailers.

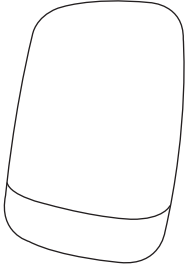
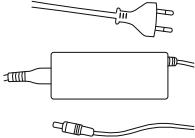
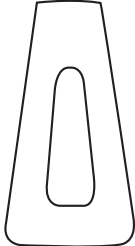
10. WHAT IF THERE ARE PROBLEMS?

Problem	Possible cause	Solution
Device does not light up	On/Off button  switched off	Switch on the On/Off button.
	No power	Connect the mains adapter correctly.
	No power	The mains adapter is faulty. Contact Customer Services or your retailer.
	LEDs have reached the end of their service life. LEDs faulty	For repairs, please contact Customer Services or an authorised retailer.

11. DISPOSAL

For environmental reasons, do not dispose of the device in household waste at the end of its service life. Dispose of the device at a suitable local collection or recycling point in your country. Observe the local regulations for material disposal. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal. You can obtain the location of collection points for old devices from the local or municipal authorities, local waste disposal company or your retailer, for example.

Component	Disposal Description	Photo
Device	The component mainly is PC and ABS. All comply with RoHS and REACH, and all could be safety disposal.	
Power adapter	The adapter mainly includes plastic and electronic components, All comply with RoHS and REACH, and all could be safety disposal.	
Accessories	All comply with RoHS and REACH, and all could be safety disposal. All comply with RoHS and REACH, and all could be safety disposal.	

12. TECHNICAL SPECIFICATIONS

Type	GCE504
Model no.	TL 85
Dimensions (L x W x H)	30 x 23.5 x 45.2 cm
Weight	approx. 1340 g
Light	LEDs
Power output	max. 60 watts
CRI	min. 90
Light intensity	10.000 lux (at a distance of approx. 25 cm)
Radiation	Radiance output outside of the visible range (infrared and UV) is low enough that no eye or skin damage is anticipated.
Operating conditions	0°C to +35°C, 15 - 90% relative humidity, 700 – 1060 hPa ambient pressure
Transport and storage conditions	-20°C to +60°C, 15 - 90% relative humidity, 700 – 1060 hPa ambient pressure
Product classification	Protection class II, IP21
Accessories	Mains adapter, instructions for use, stand
Colour temperature of the LEDs	6500K +/-400 Kelvin
Threshold of short wavelength	460 - 670 nm
Maximum light output	52.27 W / m ²
Expected service life of the device	10.000 hours

Maximum radiance output of the TL 85

Radiance output	Risk group classified in accordance with IEC 62471	Maximum value
E _{UVA} : Eye UV-A	Exempt Group	4.393e-5
ES: Actinic UV skin & eye	Exempt Group	2.794e-6
EIR: Infrared radiation hazard exposure limits for the eye	Exempt Group	4.631e-1
LIR: Retinal thermal (mild visual irritation)	Exempt Group	1.491e-1
LB: Blue light	Exempt Group	7.057e0
LR: Retinal thermal	Exempt Group	9.327e1

Subject to technical changes.

The serial number is located on the device or in the battery compartment.

Brightness: 10,000 lux (this point about the light intensity is merely for information purposes. With regard to the standard IEC 60601-2-83, this light source is classified as an Exempt Group).

Notes on electromagnetic compatibility

The device complies with Regulation (EU) 2017/745 of the European Parliament and of the Council on medical devices and the respective national regulations and the European standard EN 60601-1-2 (in accordance with CISPR 11, IEC61000-3-2, IEC61000-3-3, IEC 61000-4-2, IEC 61000-4-3, IEC 61000-4- 4, IEC 61000-4- 5, IEC 61000-4- 6, IEC 61000-4- 11, IEC 61000-4-8) and is subject to particular precautions with regard to electromagnetic compatibility.

- The device is suitable for use in all environments listed in these instructions for use, including domestic environments.
- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the display/device.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated, this device as well as the other devices must be monitored to ensure they are working properly.
- The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions or a decrease in the device's electromagnetic immunity; this can result in faulty operation.
- Failure to comply with the above can impair the performance of the device.

Mains adapter

Model no.	LXCP62 (II)-240
Input	100–240 V ~ 50/60 Hz, 1.5 A max.
Output	24 V DC, 2.5 A
Protection	The device is double-protected.
	Polarity of the DC voltage connection
Classification	IP24, protection class II

13. WARRANTY

Further information on the warranty and warranty conditions can be found in the warranty leaflet supplied.

Notification of incidents

For users/patients in the European Union and identical regulation systems (EU Medical Device Regulation (MDR) 2017/745), the following applies: If during or through use of the product a major incident occurs, notify the manufacturer and/or their representative of this as well as the respective national authority of the member state in which the user/patient is located.

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Electromagnetic compatibility

Table 1:

Guidance and manufacturer's declaration - electromagnetic emissions		
The Device is intended for use in the electromagnetic environment specified below. The customer or the user of the Device should assure that it is used in such an environment		
Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The Device is suitable for use in all establishments including domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
RF emissions CISPR 11	Class B	
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

Table 2:

Guidance and manufacturer's declaration – electromagnetic immunity			
The Device is intended for use in the electromagnetic environment specified below. The customer or the user of the Device should assure that it is used in such an environment			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±2kV ±4kV ±6kV ±8kV ±15kV	±8 kV contact ±2kV ±4kV ±8kV ±15kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %
Electrical fast transient/burst IEC 61000-4-4	±1kV, ±2kV, 100 kHz repetition frequency	Power supply lines ±2 kV	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±0.5kV, ±1kV (Line to line) ±0.5kV, ±1kV, ±2kV (Line to Ground)	Line to line: ±0.5kV, ±1kV Line to Ground: ±0.5kV, ±1kV, ±2kV	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines	0% U_T for 0.5 cycle	0% U_T for 0.5 cycle	Mains power quality should be that of a typical commercial or hospital environment.
	0% U_T for 1 cycle	0% U_T for 1 cycle	
	70% U_T	70% U_T	
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	for 25/30 cycles 0% U_T for 250/300 cycles	for 25/30 cycles 0% U_T for 250/300 cycles	Mains power quality should be that of a typical commercial or hospital environment. If the user of the model TL55 require continued operation during power mains interruptions, it is recommended that the model 168 DAYLUX daylight therapy lamp be powered from an uninterruptible power supply or a battery.

Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m 50Hz/60Hz	30 A/m 50Hz/60Hz	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE U_T is the a.c. mains voltage prior to application of the test level.			

Table 3:

Guidance and manufacturer's declaration – electromagnetic immunity			
The Device is intended for use in the electromagnetic environment specified below. The customer or the user of the Device should assure that it is used in such an environment			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC61000-4-6	150KHz to 80MHz _T 3Vrms 6Vrms (in ISM and amateur radio bands) 80% Am at 1kHz	150KHz to 80MHz _T 3Vrms 6Vrms (in ISM and amateur radio bands) 80% Am at 1kHz	Portable and mobile RF communications equipment should be used no closer to any part of the Device ,including cables, than the recommended separation distance calculated from the equation appropriate for the frequency of the transmitter. Recommended separation distances: $d=1.2\sqrt{P}$; $d=2\sqrt{P}$
Radiated RF IEC61000-4-3	10V/m	10V/m	80MHz to 800MHz _T $d=1.2\sqrt{P}$ 800MHz to 2.7GHz _T $d=2.3\sqrt{P}$ Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer, d is the recommended separation distance in meters (m) Field strengths from fixed RF transmitters, as determined by an electromagnetic site surveya ,should be less than the compliance level in each frequency range b Interference may occur in the vicinity of equipment marked with the following symbol: 

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the [XXXXX] is used exceeds the applicable RF compliance level above, the [XXXXX] should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the [XXXXX].

b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Table 4:

Recommended separation distances between portable and mobile RF communications equipment and the □				
The Device is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Device as recommended below, according to the maximum output power of the communications equipment.				
Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m			
	150 kHz to 80 MHz (out ISM and amateur radio bands) $d=1.2\sqrt{P}$	150 kHz to 80 MHz (in ISM and amateur radio bands) $d=2\sqrt{P}$	80MHz to 800MHz $d=1.2\sqrt{P}$	800MHz to 2.7GHz $d=2.3\sqrt{P}$
0.01	0.12	0.2	0.12	0.23
0.1	0.38	0.632	0.38	0.73
1	1.2	2	1.2	2.3
10	3.8	6.32	3.8	7.3
100	12	20	12	23
For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) accordable to the transmitter manufacturer.				
NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.				
NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.				

Table 5:

Guidance and manufacturer's declaration – electromagnetic immunity							
The Device is intended for use in the electromagnetic environment specified below. The customer or the user of the Device should assure that it is used in such an environment							
Radiated RF IEC61000-4-3 (Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment)	Test Frequency (MHz)	Band a) (MHz)	Service a)	Modulation b)	Modulation b) (W)	Distance (m)	IMMUNITY TEST LEVEL (V/m)
	385	380 –390	TETRA 400	Pulse modulation b) 18 Hz	1,8	0,3	27
	450	380 –390	GMRS 460,	FM c) ± 5 kHz deviation 1 kHz sine	2	0,3	28
	710	704 – 787	LTE Band 13, 17	Pulse modulation b) 217 Hz	0,2	0,3	9
	745						
	780						
	810	800 – 960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation b) 18 Hz	2	0,3	28
	870						
	930						
	1720	1 700 – 1 990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse modulation b) 217 Hz	2	0,3	28
	1845						
	1970						
	2450	2 400 – 2 570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation b) 217 Hz	2	0,3	28
	5240	5 100 – 5 800	WLAN 802.11 a/n	Pulse modulation b) 217 Hz	0,2	0,3	9
	5500						
	5785						

NOTE U_T If necessary to achieve the IMMUNITY TEST LEVEL, the distance between the transmitting antenna and the ME EQUIPMENT or ME SYSTEM may be reduced to 1 m. The 1 m test distance is permitted by IEC 61000-4-3.

- a) For some services, only the uplink frequencies are included.
- b) The carrier shall be modulated using a 50 % duty cycle square wave signal.
- c) As an alternative to FM modulation, 50 % pulse modulation at 18 Hz may be used because while it does not represent actual modulation, it would be worst case.

The MANUFACTURER should consider reducing the minimum separation distance, based on RISK MANAGEMENT, and using higher IMMUNITY TEST LEVELS that are appropriate for the reduced minimum separation distance. Minimum separation distances for higher IMMUNITY TEST LEVELS shall be calculated using the following equation:

$$E=6/d \sqrt{P}$$

Where P is the maximum power in W, d is the minimum separation distance in m, and E is the IMMUNITY TEST LEVEL in V/m.

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Globalcare Medical Technology Co., Ltd.
 7th Building, 39 Middle Industrial Main Road, European Industrial Zone,
 Xiaolan Town, 528415
 Zhongshan City, Guangdong Province, P.R. China
 Phone : +86 760 22589901

BEURER GmbH • Söflinger Str. 218 • 89077 Ulm (Germany)
 Tel.: +49 (0) 731 / 39 89-144 • Fax: +49 (0) 731 / 39 89-255
 www.beurer.com • Mail: kd@beurer.de

Donawa Lifescience, Piazza Albania, 10, 00153 Rome, Italy
<https://www.donawa.com/wli/main/contatti.index>

Camara and Partners Sàrl, Route de St Cergue 14, 1260 Nyon, Switzerland
 Tel: +41 22 362 48 13