DE	Massagesessel Gebrauchsanweisung	2
EN	Massage chair Instructions for use	12
FR	Fauteuil de massage Mode d'emploi	22
ES	Sillón de masaje Manual de instrucciones	32

IT	Poltrona per massaggi Istruzioni per l'uso	42
TR	Masaj koltuğu Kullanım kılavuzu	52
RU	Массажное кресло Инструкция по применению	62
PL	Fotel do masażu Instrukcja obsługi	72

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Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Contents

1.Intended use.....	13	6. Maintenance and storage	19
2.Safety notes.....	13	7. What if there are problems.....	19
3.Unit description	15	8. Technical Data.....	20
4.Initial use	17	9. Disposal	20
5.Operation.....	18	10. Warranty and service	20

Included in delivery

- Massage chair
- Back cushion
- Hand switch
- Seat cushion
- Pillow
- Battery set

⚠ WARNING

- The unit is only intended for domestic/private use, not for commercial use.
- This device may be used by children over the age of 8 and by people with reduced physical, sensory or mental skills or a lack of experience or knowledge, provided that they are supervised or have been instructed on how to use the device safely, and are fully aware of the consequent risks of use.
- Children must not play with the device.
- Cleaning and user maintenance must not be performed by children unless supervised.
- If the mains connection cable of this device is damaged, it must be disposed of. If it cannot be removed, the device must be disposed of.
- Do not insert any objects into the device apertures or the rotating parts. Ensure that the moving parts can move freely at all times.
- Under no circumstances should you open or repair the device yourself, as faultless functionality can no longer be guaranteed thereafter. Failure to adhere to this instruction will void the warranty.

Signs and symbols

The following symbols appear in the instructions for use and on the type plate:

	WARNING Warning instruction indicating a risk of injury or damage to health.
	IMPORTANT Safety note regarding potential for damage to the device/accessories
	Note Note on important information.
	Only use indoors.
	Do not insert needles!
	CE labelling This product satisfies the requirements of the applicable European and national directives.
	The products demonstrably meet the requirements of the Technical Regulations of the EAEU.

	Please dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment).
	Manufacturer
	United Kingdom Conformity Assessed Mark

1. Intended use

The massage chair is intended for the purpose of massaging the back and legs. Please observe the safety notes in these instructions for use. Use the device only according to the methods specified. The device is suitable only for private use and not for medical or commercial purposes.

2. Safety notes

Electric shock

Warning!

- As with every electrical device, use this massage chair carefully and cautiously in order to avoid hazards resulting from electric shock.
- Use the device only with the mains voltage specified on the device.
- Never use the device if it or any of the accessories show visible damage.
- Do not use the device if it was accidentally splashed with water.
- Do not use the device during a storm.
- Only use in closed rooms and avoid direct sunlight.
- In the event of defects or operational faults, switch off the device immediately. Pull the mains plug out of the socket.
- Never pull on the mains cable to remove the mains plug from the socket.
- Do not clamp or pinch the mains cable.
- Keep the mains cable away from hot surfaces.
- Ensure that the device, switch and cable do not come into contact with water, steam or other liquids.
- Use/operate the device only with dry hands.
- Do not subject the device to any impacts or drop the device.

Repairs

Warning!

- Do not open the device.
- Only specialist personnel may perform repairs on electrical devices. Improper repairs may subject users to considerable danger. For repairs, please contact Customer Service or an authorised retailer. Failure to do so invalidates the warranty.
- The zip fastening beneath the cover must not be opened. It is only used for technical production reasons.
- Do not insert needles or other sharp objects into the chair.

Risk of fire

Warning!

There may be a risk of fire if the device is used improperly or these instructions for use are ignored!

- Ensure that the vents are free at all times. Always keep the vents free from dust, hair etc.
- Do not use the device together with other electrical devices (e.g. electric blanket).
- Never use the device near petrol or other highly flammable substances.

Operation

Warning!

This device is exclusively intended to massage the back and legs of humans.

It cannot replace medical treatment. Do not use the massager if one or several of the following warnings applies to you.

Do not use the massager

- if you suffer from a medical abnormality or injury to your back (e.g. slipped disc)
- During pregnancy
- While sleeping
- On animals
- Whilst undertaking any activity where an unexpected reaction could be dangerous
- After the consumption of substances that lead to limited cognitive ability (e.g. pain medication, alcohol)

If you are unsure whether the massager is suitable for you, please consult your doctor.

You should consult your doctor before using the massager


- If you suffer from a severe illness or recently had surgery on your upper body
- If you have a pacemaker, implant or other aids
- Thromboses
- Diabetes
- If you are suffering pain and the cause of which is unknown.
- The device is only intended for the purpose described in these instructions for use. The manufacturer is not liable for damage resulting from improper or careless use.
- Do not touch the massage mechanism or the rollers with your hands during operation; there is a danger of crushing.
- Do not insert any objects in the device apertures or in the rotating parts. Ensure that the moving parts can move freely at all times.
- Always switch off the device at the main switch after use.
- Ensure that the mains cable does not pose a trip hazard.
- Never stand on the device or place any objects on the device.
- Never massage areas that exhibit swellings, infections or eczema.
- Do not use the device in the event of inexplicable stomach ache.
- Only one person may use the massage chair at any given time.
- Never load the device with more than 150 kg.

Caution!

- Do not expose the device to direct sunlight.
- Do not use the device at temperatures above 40°C.
- Check the function of the device if it has not been used for a longer period of time.

3. Unit description

External features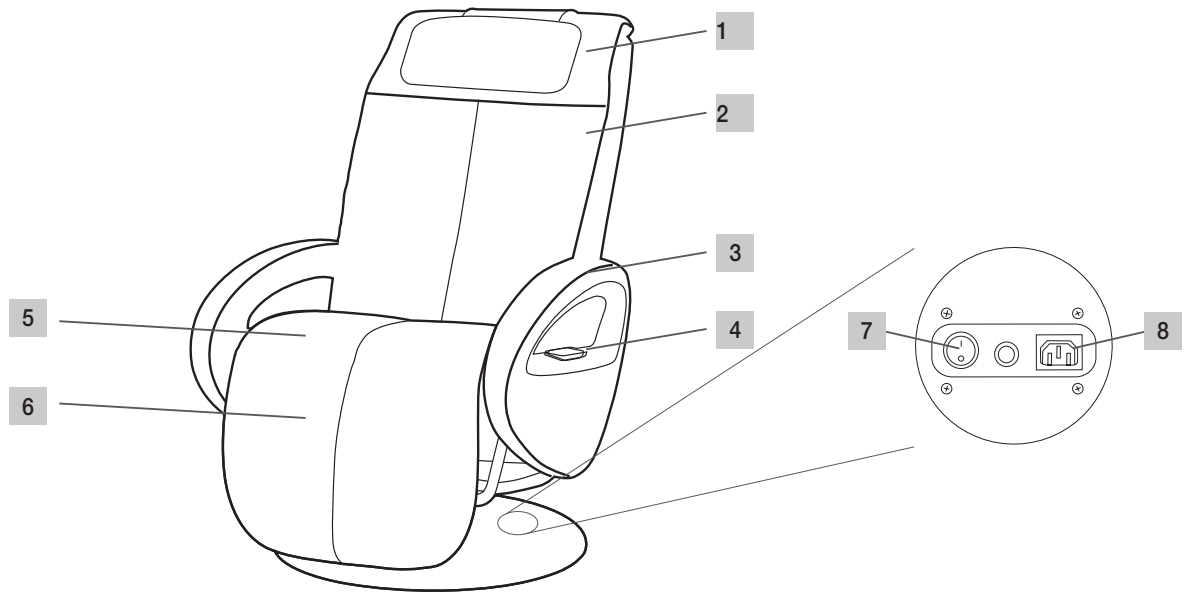

1 Pillow

4 Hand switch with battery compartment

7 Main switch

2 Back rest


5 Seat cushion

8 Mains socket

3 Arm rest

6 Leg rest


Internal features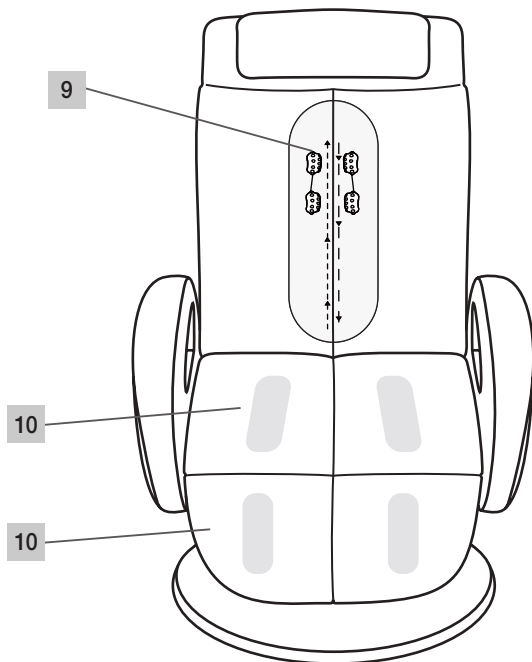

9 Massage rollers

10 Vibration massage points

Hand switch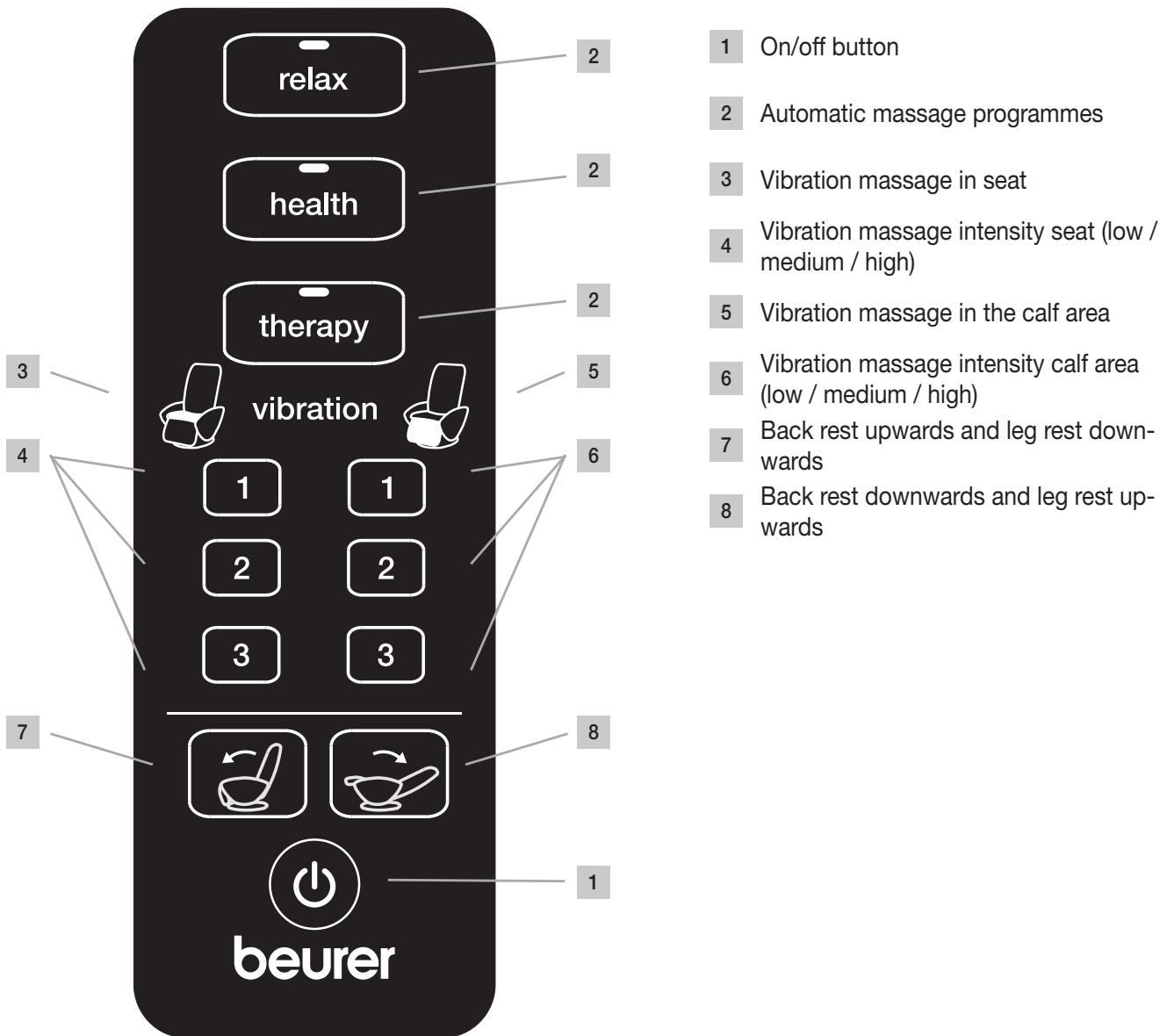

Hand switch button functions

Automatic massage programmes

There are three automatic massage programmes – **relax**, **health** and **therapy**.

These automatic programmes massage your entire back with a combination of kneading, tapping, rolling and Shiatsu massage elements.

relax

> Press the **relax** button

The automatic massage programme starts.

Total duration: 15 minutes. The massage chair then remains in the position set by you.

health

> Press the **health** button

The automatic massage programme starts.

Total duration: 15 minutes. The massage chair then remains in the position set by you.

therapy

> Press the **therapy** button

The automatic massage programme starts.

Total duration: 15 minutes. The massage chair then remains in the position set by you.

i Note

The vibration massage functions independently of the massage programmes.
To allow this, the On/Off button must be activated and the current massage programme deactivated.

Vibration massage / deactivation

You can deactivate the vibration massage by pressing the intensity button again.
Pressing the intensity button once activates the vibration massage; pressing the same button again deactivates it.

4. Initial use

⚠ Warning!

- Keep packaging material away from children!
There is a choking hazard.
- Before using the device for the first time, read the safety notes in the Safety section.
 - ▷ Safety, pages 13 to 15

⚠ Caution!

- The massage chair could damage your flooring. Place a suitable, non-slip underlay under the massage chair.


Installation location

- Install the massage chair on a sturdy and even surface.
- Before using the device for the first time, remove all obstacles in front of and behind the massage chair so that the back rest can be lowered and the leg rest raised freely.


⚠ Caution!

Do not use sharp or pointed tools (Stanley knife) to remove the packaging.


- 1.) Insert 2 x AAA batteries in the hand switch according to the markings.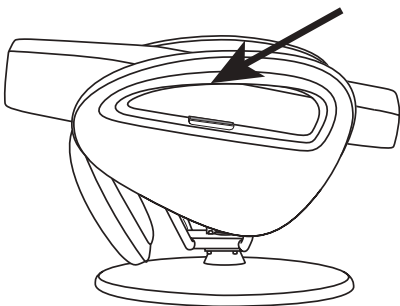


- 2.) Lift the massage chair out of the box using the designated holding points. Two people are required.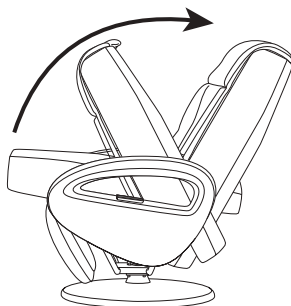

- 3.) Unfold the back rest to the upright position until the locking mechanism audibly engages.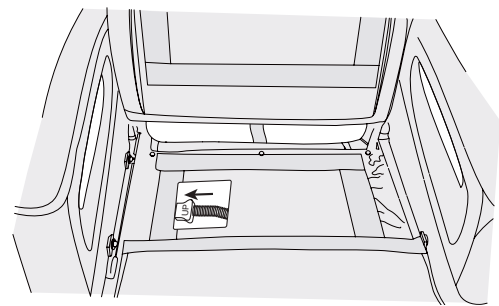

- 4.) Insert the plug attached to a spiral spring into the socket provided. Then fold the seat cushion against the back rest.

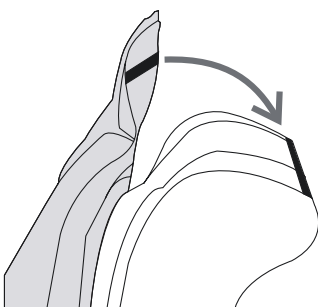- 5.) Now connect the back cushion to the seat cushion and the back rest.

- 6.) The massage chair is now ready for operation.

5. Operation

Warning!

- Before using the device for the first time, read the safety notes in the Safety section.
 - ▷ Safety, pages 13 to 14

Danger of tipping!

- Sit down carefully on the chair and only when it is in the upright position. Never sit down on the chair when it is in the flat position.

Caution!

- If you experience irritations of any kind, stop treatment immediately.

Important!

- Never remove the back cushion during the massage.

Before switching on

- Wear comfortable clothes during the massage.
- Remove all objects from your trouser pockets (e.g. keys, pocket knife, mobile phone).
- Assume a comfortable, relaxed position.
- Always sit in the middle of the seat so that your spine is positioned precisely between the massage rollers.
- The massage should feel pleasant and relaxing at all times. If the massage is painful or uncomfortable, stop the massage or change your sitting position.
- If the device has not been used for a longer period of time, check that the device is functioning correctly before use.


Starting the massage

Press the on/off button on the hand switch.

After being switched on, the back rest reclines and the leg rest is raised. At the same time, the entire back is scanned to determine the size of the user. The **relax** massage programme is activated after the user's back has been scanned.

- Press the vibration intensity button in order to adjust or switch off the intensity of the vibration massage in the seat or foot area.
- The vibration massage can be switched on or off separately for the seat and feet.

Flat position/upright position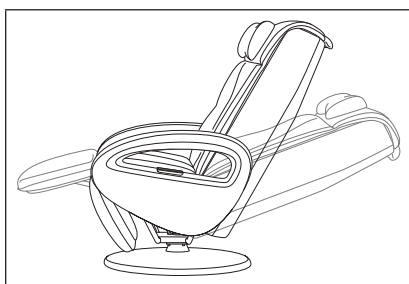

- Put the massage chair in the flat position.

Switching off the massage chair

After a massage programme is finished, the chair automatically switches off. The current position of the massage chair is maintained. By pressing the On/Off button again, the relax massage programme is restarted. The massage chair returns to the start position if the On/Off button is pressed twice.

Caution!


Do not use the device for longer than 15 minutes at a time otherwise the device could overheat. Allow the device to cool for at least 30 minutes before using it again.

During the massage, you can switch off the device at any time using the on/off button.

- Switch off the massage chair using the main switch beneath the back rest.
- Pull the mains plug out of the socket

Additional settings, operating options

Folding mechanism

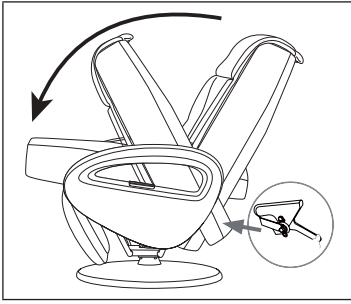The massage chair can be folded away to save space when stored. To do this, follow the process for initial use in section 5 in reverse order.

6. Maintenance and storage

Warning!

- The device must be switched off at the main switch after every use and before cleaning.
- Pull the mains plug out of the socket.
- Wipe the massage chair from time to time with a dry, lint-free or slightly damp cloth.

Important!

- Clean the device using only the methods specified.
- Under no circumstances may liquid enter the device.
- Do not use any solvent-based cleaning products.
- Do not use aggressive cleaning products. These can damage the surface.

Artificial leather is sensitive to grease. Clean the massage chair only using care products designated for artificial leather.

7. What if there are problems

Problem	Cause	Solution
Grinding noises during operation.	These noises are normal and originate from the operating motor or from the massage rollers rubbing against the cushion.	No action is necessary.
The massage cannot be started.	The mains plug is not connected to the socket.	Plug the mains plug into the socket.
	The massage chair is not switched on at the main switch.	Switch on the massage chair at the main switch.
	The cutout switch for the massage chair has interrupted the power supply.	In this case, please contact Customer Service.
Back rest and leg rest cannot be raised or lowered.	Back rest and leg rest are overloaded or blocked.	Remove the objects blocking the device.
The massage function stops when the chair is in operation.	The overheating protection was triggered.	Switch off the massage chair and allow to cool down for 30 minutes before switching on again.
	The maximum operation period was reached.	

Problem	Cause	Solution
Hand switch does not work.	Hand switch is not configured to the massage chair.	To configure the hand switch, point it towards the bottom half of the back rest. It should be no more than 40 cm away. Press the On/Off button on the hand switch until the LED above the relax massage programme lights up. After approximately 3-4 seconds, LED 4 starts to flash and then switches off. The hand switch is now configured and ready for use.
Hand switch/massage chair does not react when several devices are connected.	Hand switch signal is not uniquely assigned to one device	Maintain a minimum distance of approx. 15 metres between the massage chairs.

8. Technical Data

Supply	220-240 V~, 50/60 Hz, 120 W
Dimensions	162 x 73 x 76 cm
Weight	ca. 50,5 kg
Maximum load capacity	150 kg

9. Disposal

Empty, completely flat batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries.

The codes below are printed on batteries containing harmful substances:

Pb = Battery contains lead,

Cd = Battery contains cadmium,

Hg = Battery contains mercury.

For environmental reasons, do not dispose of the device in the household waste at the end of its service life. Dispose of the device at a suitable local collection or recycling point.

Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment).

If you have any questions, please contact the local authorities responsible for waste disposal.

10. Warranty and service

Beurer GmbH, Söflinger Straße 218, 89077 Ulm, Germany (hereinafter referred to as “Beurer”) provides a warranty for this product, subject to the requirements below and to the extent described as follows.

The warranty conditions below shall not affect the seller’s statutory warranty obligations which ensue from the sales agreement with the buyer.

The warranty shall apply without prejudice to any mandatory statutory provisions on liability.

Beurer guarantees the perfect functionality and completeness of this product.

The worldwide warranty period is 2 years, commencing from the purchase of the new, unused product from the seller.

The warranty only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use.

German law shall apply.

During the warranty period, should this product prove to be incomplete or defective in functionality in accordance with the following provisions, Beurer shall carry out a repair or a replacement delivery free of charge, in accordance with these warranty conditions.

If the buyer wishes to make a warranty claim, they should approach their local retailer in the first instance: see the attached “International Service” list of service addresses.

The buyer will then receive further information about the processing of the warranty claim, e.g. where they can send the product and what documentation is required.

A warranty claim shall only be considered if the buyer can provide Beurer, or an authorised Beurer partner, with

- a copy of the invoice/purchase receipt, and
- the original product.

The following are explicitly excluded from this warranty:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);
- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions of the instructions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by Beurer;
- damage that arises during transport between manufacturer and customer, or between service centre and customer;
- products purchased as seconds or as used goods;
- consequential damage arising from a fault in this product (however, in this case, claims may exist arising from product liability or other compulsory statutory liability provisions).

Repairs or an exchange in full do not extend the warranty period under any circumstances.

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