beurer

stress releaZer®

DE	stress releaZer Gebrauchsanweisung.....	2
EN	stress releaZer Instructions for use.....	17
FR	stress releaZer Mode d'emploi	30
ES	stress releaZer Instrucciones de uso	44
IT	stress releaZer Istruzioni per l'uso	58
TR	stress releaZer Kullanım kılavuzu.....	72
RU	stress releaZer Инструкция по применению	86
PL	stress releaZer Instrukcja obsługi	99

CE

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Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

WARNING

- The device is intended only for domestic/private use, not for commercial use.
- This device can be used by children over the age of 8 and by people with reduced physical, sensory or mental skills or a lack of experience or knowledge, provided that they are supervised or have been instructed on how to use the device safely and are aware of the consequent risks of use.
- Children must not play with the device.
- Cleaning and user maintenance must not be performed by children unless supervised.
- If the mains connection cable of this device is damaged, it must be disposed of. If it cannot be removed, the device must be disposed of.
- Under no circumstances should you open or repair the device yourself, as faultless functionality can no longer be guaranteed thereafter. Failure to adhere to this instruction will void the warranty.

Dear customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, gentle therapy, blood pressure/diagnosis, weight, massage, beauty, air and baby-care.

With kind regards,
Your Beurer team

Table of contents

1. Getting to know your device	20	6.1 Positioning the stress releaZer .	25
1.1 How does the stress releaZer work?.....	20	6.2 Switching on the stress releaZer/ selecting the breathing cycle....	25
1.2 "beurer CalmDown" app	20	6.3 Heat function	26
2. Intended use	21	6.4 Light function	26
3. Notes	21	7. Cleaning and maintenance	26
4. Device description	23	8. Disposal	27
5. Initial use	24	9. Technical specifications	27
6. Use	25	10. Warranty / service	28


Included in delivery

1x stress releaZer
1x micro USB charging cable
1x mains adapter

Signs and symbols

The following symbols appear in these instructions for use:

	WARNING Warning notice indicating a risk of injury or damage to health
	IMPORTANT Safety note indicating possible damage to the device/accessory
	Note Note on important information

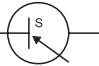	Observe the instructions for use
	Device not to be used on individuals with a pacemaker or other implants, such as an insulin pump or an implanted defibrillator
	The device is double protected and therefore corresponds to protection class 2.
	Only use indoors.
	Manufacturer
	Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE
	Dispose of packaging in an environmentally friendly manner
	Certification symbol for products that are exported to the Russian Federation and members of the CIS
	Direct current
	Energy efficiency class 6
	Switching mains part
	Short-circuit-proof safety isolating transformer

1. Getting to know your device

1.1 How does the stress releaZer work?

The beurer stress releaZer was specially developed for the prevention and treatment of stress. By covering the diaphragm area with the stress releaZer, primary nerve pathways in this region are relaxed and your natural breathing rhythm is restored.

The built-in vibration motor transmits pulses to the user, which allows them to breathe normally. The stress releaZer also has an illuminated ring, which functions as a mood light. While the device is in use, the illuminated ring pulsates in time with the user's breathing rhythm.

The stress releaZer also has an optional heat function on the underside for a more comfortable application.

1.2 “beurer CalmDown” app

You can also connect the stress releaZer to your smartphone using *Bluetooth*[®]. To do this, all you need to do is download the free “beurer CalmDown” app from the Apple App Store (iOS) or from Google Play (Android[™]).

The free “beurer CalmDown” app offers the following advantages:

- Detailed instructions for use with videos
- Audiovisual breathing exercises with an HRV (heart rate variability) training programme
- Relaxing music to accompany use
- A calendar function for recording periods of use
- An option to add personal notes

- System requirements:

iOS ≥ 10.0

Android[™] ≥ 5.0

Bluetooth[®] ≥ 4.0

- Data transfer via *Bluetooth*[®] wireless technology.

The stress releaZer[®] uses *Bluetooth*[®] low energy technology, frequency band 2.400 - 2.480 GHz, the maximum transmission power emitted in the frequency band < 20 dBm, compatible with *Bluetooth*[®] 4.0 smartphones/tablets.

Technical information is subject to change without notification to allow for updates.

2. Intended use

WARNING

The stress releaZer is intended for private use only. The stress releaZer is not a medical device, rather it is a relaxation device. The stress releaZer may only be used for the purpose for which it is designed and in the manner specified in the instructions for use. Any form of improper use can be dangerous. The manufacturer is not liable for damage resulting from improper or careless use.

3. Notes

WARNING

Safety note regarding the use of the stress releaZer

- Keep the packaging material away from children. choking hazard!
- Before use, ensure that the device and accessories have no visible damage. If you have any doubts, do not use the device and contact your retailer or the specified Customer Service address.
- Keep the mains adapter away from water.
- Do NOT use the device if you have a pacemaker or other implants, such as an insulin pump or an implanted defibrillator.
- Do NOT use the device on the head.
- Do NOT use the device on the face.
- Do NOT use the device on the genitals.
- Do NOT use the device if you suffer from a medical abnormality or injury in the diaphragm area (e.g. open wound).
- Do NOT use the device during pregnancy.
- Do NOT use the device while sleeping.
- Do NOT use the device in a vehicle.
- Do NOT use the device on animals.
- Do NOT use the device whilst undertaking any activity where an unexpected reaction could be dangerous.
- Do NOT use the device after the consumption of substances that lead to impaired cognitive function (e.g. pain medication, alcohol).
- The device features an optional heat function. Persons not sensitive to heat must be careful when using the heat function.
- Use of the device must feel pleasant at all times.

- Only use the device with the supplied accessories.
- Repairs must only be carried out by Customer Services or authorised retailers. Under no circumstances should you open or repair the device yourself, as faultless functionality could no longer be guaranteed thereafter. Failure to comply with this instruction will void the warranty. Never attempt to repair the device yourself!

If you are unsure whether the device is suitable for you, please consult your doctor.


Please consult your doctor before using the stress releaZer:

- If you suffer from a severe illness or have recently had surgery on your upper body.
- In the case of feverish colds, thrombosis, diabetes, neurological diseases or severe inflammations.
- If you are suffering pain, the cause of which is unknown.

Notes on handling batteries

- If your skin or eyes come into contact with fluid from a rechargeable battery cell, flush out the affected areas with water and seek medical assistance.
- Choking hazard! Small children may swallow and choke on rechargeable batteries. Store rechargeable batteries out of the reach of small children.
- Protect batteries from excessive heat.
- Risk of explosion! Never throw batteries into a fire.
- Do not disassemble, split or crush the rechargeable batteries.
- Only use chargers specified in the instructions for use.
- Batteries must be charged correctly prior to use. The instructions from the manufacturer and the specifications in these instructions for use regarding correct charging must be observed at all times.
- Fully charge the battery prior to initial use.
- In order to achieve as long a battery service life as possible, fully charge the battery at least twice per year.

4. Device description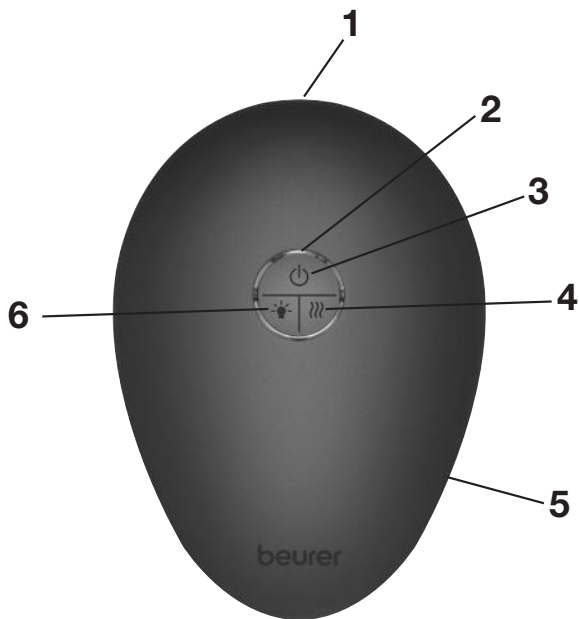

1 Micro USB connection

4 Heat button

2 Function light

5 Illuminated ring

3 ON/OFF button ⏻

6 Illuminated ring button

5. Initial use

Note

Before using the stress releaZer for the first time, let it charge for a minimum of 4 hours. Proceed as follows:

Charging the stress releaZer

- Connect the micro USB charging cable to the mains adapter and the stress releaZer (Figure 1). Then insert the mains adapter into a suitable socket. Alternatively, you can also charge the stress releaZer with the charging cable via the USB port on your computer or laptop (Figure 2).

When the device is charging, the function light will light up alternately in a circle. When the stress releaZer is fully charged, the function light will be permanently illuminated. The battery will last for approx. 3 hours when fully charged (with heat and light function activated) or approx. 11 hours without heat and light function. Charge the stress releaZer again as soon as the function light flashes blue during operation.

Note

The device cannot be used whilst it is being charged.

Figure 1

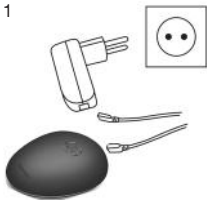
Figure 2

Further information and FAQs available at:
<http://www.beurer.com>


6. Use

6.1 Positioning the stress releaZer

Note

You can use the stress releaZer either on bare skin or over clothing. We recommend that you use the stress releaZer for 30 minutes every day.

1. Lie flat on your back. Alternatively, you can also sit on a reclined chair.
2. Place the stress releaZer on the diaphragm area (resting on the bottom rib bones/chest). If you find that use directly on the diaphragm is uncomfortable, position the stress releaZer above the diaphragm.

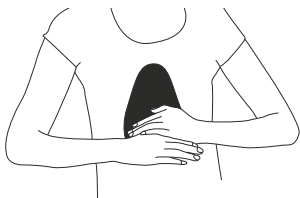Note

Do not use the device near your solar plexus (area between your chest and your stomach). This may cause dizziness and nausea.

6.2 Switching on the stress releaZer/selecting the breathing cycle

Note

Select the breathing cycle of your choice. If the stress releaZer is used over an extended period of time, it is possible to increase your breath duration.

Press  once	=	Press the ON/OFF button  to switch on the stress releaZer. <i>Bluetooth</i> [®] is activated when three sections of the function light light up blue. You can now connect the stress releaZer to your smartphone using the “beurer CalmDown” app if you wish.
Press  twice	=	The first breathing cycle is activated. In this breathing cycle, breathe in for 5 seconds, and then breathe out for 5 seconds.
Press  three times	=	The second breathing cycle is activated. In this breathing cycle, breathe in for 6 seconds, and then breathe out for 6 seconds.

Press  four times	=	The third breathing cycle is activated. In this breathing cycle, breathe in for 7 seconds, and then breathe out for 7 seconds.
Press  five times	=	The stress releaZer is switched off.

Note

You can also switch off the stress releaZer, regardless of the setting, by pressing and holding down the ON/OFF button.

Note

The stress releaZer features an automatic switch-off function, which is set to the maximum operating duration of 30 minutes. After 30 minutes, the stress releaZer switches off automatically.

6.3 Heat function

You can switch on a soothing heat during use to increase relaxation. To do this, press the heat button  on the stress releaZer. The function light starts to glow blue. Press the heat button  again to switch off the heat.

6.4 Light function

You can also switch on a blue light during use to increase relaxation. To do this, press the light button  on the stress releaZer. The illuminated ring around the stress releaZer will now start to pulse in time with the user's breathing rhythm. Press the light button  again to switch off the blue light. When the light function and the heat function are both switched on, the illuminated ring will glow red, not blue.

7. Cleaning and maintenance

The service life of the stress releaZer depends on careful use.

IMPORTANT

- Clean the stress releaZer after every use using a soft, slightly damp cloth (water or a mild cleaning solution). Dry the stress releaZer using a lint-free cloth. Do not use any abrasive cleaning products.

- Protect the stress releaZer from knocks, dust, chemicals, drastic changes in temperature, electromagnetic fields and nearby sources of heat (ovens, heaters).
- Do not submerge the stress releaZer in water or any other fluids and make sure that no fluids can get into the stress releaZer.

8. Disposal

For environmental reasons, do not dispose of the device in the household waste at the end of its useful life.

Dispose of the device at a suitable local collection or recycling point in your country. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

Rechargeable battery disposal

The empty, completely flat rechargeable batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries.

To remove the battery from the device, pry open the cover above the illuminated ring using a screwdriver. Remove the battery from inside the housing.

The codes below are printed on rechargeable batteries containing harmful substances:

Pb = Battery contains lead

Cd = Battery contains cadmium

Hg = Battery contains mercury

9. Technical specifications

Supply	100-240 V~; 50/60 Hz; 0.2 A
Dimensions	18 x 12.8 x 5 cm
Weight	Approx. 380 g
<u>Battery:</u>	
Capacity	2200 mAh
Nominal voltage	3.7 V
Type designation	Lithium-ion

Data transfer	<i>Bluetooth</i> [®] low energy technology, 2.400 - 2.480 GHz frequency band transmission power < 20 dBm 
---------------	---

Technical information is subject to change without notification to allow for updates.

We hereby guarantee that this product complies with the European RED Directive 2014/53/EU. Please contact kd@beurer.de to obtain detailed information, such as the CE Declaration of Conformity.

10. Warranty / service

Beurer GmbH, Söflinger Straße 218, 89077 Ulm, Germany (hereinafter referred to as “Beurer”) provides a warranty for this product, subject to the requirements below and to the extent described as follows.

The warranty conditions below shall not affect the seller’s statutory warranty obligations which ensue from the sales agreement with the buyer.

The warranty shall apply without prejudice to any mandatory statutory provisions on liability.

Beurer guarantees the perfect functionality and completeness of this product.

The worldwide warranty period is 3 years, commencing from the purchase of the new, unused product from the seller.

The warranty only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use.

German law shall apply.

During the warranty period, should this product prove to be incomplete or defective in functionality in accordance with the following provisions,

Beurer shall carry out a repair or a replacement delivery free of charge, in accordance with these warranty conditions.

If the buyer wishes to make a warranty claim, they should approach their local retailer in the first instance: see the attached “International Service” list of service addresses.

The buyer will then receive further information about the processing of the warranty claim, e.g. where they can send the product and what documentation is required.

A warranty claim shall only be considered if the buyer can provide Beurer, or an authorised Beurer partner, with

- a copy of the invoice/purchase receipt, and
- the original product.

The following are explicitly excluded from this warranty:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);
- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions of the instructions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by Beurer;
- damage that arises during transport between manufacturer and customer, or between service centre and customer;
- products purchased as seconds or as used goods;
- consequential damage arising from a fault in this product (however, in this case, claims may exist arising from product liability or other compulsory statutory liability provisions).

Repairs or an exchange in full do not extend the warranty period under any circumstances.

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The *Bluetooth*[®] word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Beurer GmbH is under license. Other trademarks and trade names are those of their respective owners.

Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries.

Google Play and the Google Play logo are trademarks of Google LLC.

Android is a trademark of Google LLC.

Beurer GmbH • Söflinger Str. 218 • 89077 Ulm, Germany
www.beurer.com • www.beurer-healthguide.com
www.beurer-gesundheitsratgeber.com