**READ THIS MANUAL COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT**

Keep this manual in a safe location for future reference

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## CONTENT

1. Getting to know your instrument .....	2	7. Evaluating Results .....	7
2. IMPORTANT SAFETY INSTRUCTIONS.....	2	8. Care and Maintenance .....	8
3. Parts and Controls.....	5	9. Troubleshooting.....	8
4. Initial Use .....	5	10. Technical specifications.....	9
5. Entering User Data .....	6	11. FCC Part 15, Subpart B.....	9
6. Taking Measurements .....	6	12. Warranty .....	9

## SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE

If the instruction manual is damaged or if you no longer have the instruction manual in your possession, please contact Beurer customer service. Please see warranty for service contact.

### Dear Customer,

Thank you for choosing this Beurer product. Our name stands for high-quality, thoroughly tested products for applications in the areas of heating, weight management, blood pressure, body temperature, pulse, gentle therapy, massage, and beauty.

Please read these instructions for use carefully and keep them for later use. Make them accessible to other users and observe the information they contain.

With kind regards,  
Your Beurer team

### Included in delivery

- Body analysis scale BF 221
- 2 x 3.0 V batteries, CR 2032
- Quick start guide
- Instruction manual

## 1. Getting to know your instrument

### Functions of the device

This digital diagnostic scale is intended for measuring body weight. It is intended for household use only. In addition to measuring body weight, the scale has the following functions that can be used by up to ten persons:

- Weight (lb, kg or st)
- Body fat percentage BF in %
- Water percentage  $\approx$  in %
- Muscle percentage  $\text{■}$  in %

### The scale also includes the following features:

- Switches between pounds “lb”, kilograms “kg” and stones “st”
- Automatic shut-off
- Low battery indicator

## 2. IMPORTANT SAFETY INSTRUCTIONS

### Signs and symbols

The following signs appear in the Safety Section (page 3) and in this manual on page 3.

**READ AND UNDERSTAND THIS ENTIRE MANUAL INCLUDING THE SAFETY SECTION AND ALL INSTRUCTIONS AND WARNINGS COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT. FOLLOW ALL SAFETY INSTRUCTIONS AND WARNINGS TO AVOID HAZARDOUS SITUATIONS AND TO MAKE CORRECT USE OF THIS PRODUCT.**

	This is the safety alert symbol. It alerts you to potential personal injury hazards. Obey all safety messages that follow this symbol to avoid possible injury or death.
 <b>WARNING</b>	<b>WARNING</b> indicates a hazardous situation which, if not avoided, could result in death or serious injury.
 <b>CAUTION</b>	<b>CAUTION</b> indicates a hazardous situation which, if not avoided, may result in minor or moderate injury.
<b>NOTICE</b>	<b>NOTICE</b> addresses practices not related to personal injury, such as product and/or property damage.
	Type BF applied part
	Protect from moisture
	Observe the instructions for use

-  **WARNING – To reduce the risk of fire, electric shock, or serious personal injury:**
- The scale must not be used by people with medical implants (e.g. pacemakers), as this may affect their functionality.
  - Do not use during pregnancy.
  - To prevent falls, do not step onto the scale with wet feet or if the surface of the scale is damp.

-  **CAUTION – To reduce the risk of personal injury or product/property damage:**
- This product is intended solely for household use and not for medical or commercial purposes.
  - Not to diagnose or treat any medical condition.
  - Place the scale on a hard, flat, level surface; a hard surface is crucial to accurate measurements.
  - Protect the device from impacts, moisture, dust, chemicals, large temperature fluctuations, and nearby sources of heat (ovens, heaters).

**NOTICE:**

- The device should be cleaned from time to time. Do not use any abrasive cleaning products and never submerge the device in water.
- Measurement variations are normal, as this scale is not calibrated for use in a professional medical environment.
- Make sure that no liquids come into contact with the scale. Never submerge the scale in water. Never rinse it in running water.
- Do not place any objects on the scale when not in use.
- Do not press the buttons violently or with pointed objects.
- Do not expose the scale to high temperatures or strong electromagnetic fields.

**General Notes:**

- The capacity of the scale is 400 lb (180 kg, 28 st).
- For weight measurement, the results are shown in 0.2 lb increments (100 g, 0.01 st).
- The measuring results of the body fat, body water and muscle percentage are shown in 0.1 % increments.
- Ages 10 to 100 and height settings from 3'03" to 7'03" (100 to 220 cm) can be preset.

- On delivery, the scale is set to “inch” and “lb”. To the rear of the scale, there is a button where you can select “pound”, “stone” and “kilogram” (lb, st, kg).
- Repairs must only be performed by Beurer Customer Service.

## **Introduction**

This scale works on the principle of Bioelectrical Impedance Analysis (BIA). This involves a quick calculation of body content using a harmless electric current that is not even noticeable.

When measurement of the electrical resistance (impedance) is considered alongside other data (age, height, gender, and activity level), body fat percentage and other variables in the body can be calculated. Muscle tissue and water conduct electricity well and therefore have a lower resistance. In contrast, bones and fat tissue have low conductivity as the fat cells and bones are barely able to conduct the current due to very high levels of resistance.

Please note that the values calculated by the analysis scale only represent an approximation of the actual values for your body. Only a doctor can precisely establish body fat, body water, muscle percentage and bone structure using medical methods.

## **General tips**

- If possible, always weigh yourself at the same time every day (ideally in the morning), after having used the restroom, on an empty stomach and with as little clothing as possible, in order to obtain comparable, consistent results.
- Calculation of body fat may only be made when barefoot; the soles of the feet may be lightly dampened where appropriate. Results may be unsatisfactory if the soles of the feet are completely dry or feature high levels of the horny layer of the skin as this leads to insufficient conductivity.
- Stand up straight and still while weighing.
- Wait a few hours after significant physical exertion.
- Wait about 15 minutes after awakening so that your body's water content can normalize.
- Remember that only long-term trends are important. Brief deviations in weight within a few days are normally caused by loss of fluids; however, body water plays an important role in terms of well-being.

## **Limitations**

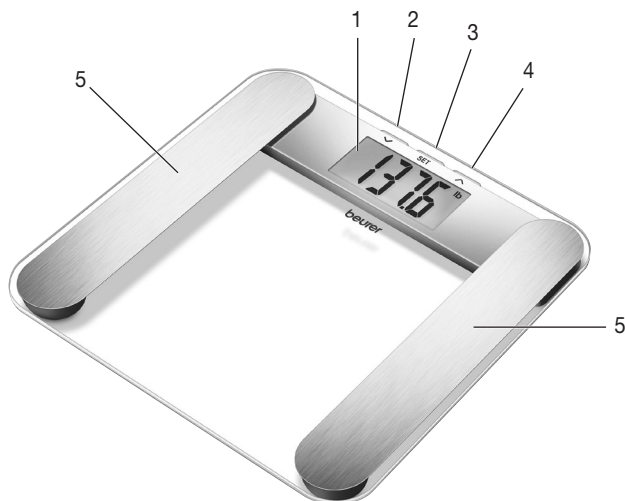
When calculating body fat and additional values, deviating and unexpected results may occur in the case of:

- Children under 10 years of age
- Performance athletes and bodybuilders
- Pregnant women
- Persons with a fever, persons undergoing dialysis, persons demonstrating edema symptoms, and persons suffering from osteoporosis
- Persons taking cardiovascular medicine (affecting the heart and vascular system)
- Persons taking vasodilating or vasoconstrictor drugs
- Persons with significant anatomical deviations in their legs compared with their overall body height (leg length significantly shorter or longer).

### 3. Parts and Controls

#### Overview

1. Display
2. "Down" key ▼
3. "SET" key
4. "UP" key ▲
5. Conductors

### 4. Initial Use

#### Inserting/replacing batteries

Remove the plastic battery isolator strip by pulling it from the Battery Compartment Cover. If the scale does not function, remove the batteries completely and reinsert them, ensuring proper polarity. When battery power becomes low, "LO" will appear in the LCD Display and the scale may shut off. Replace both batteries with two new CR2032 button cell batteries, observing the correct polarity.

#### Battery Handling Safety Precautions

- Use only the size and type of batteries specified.
- Be sure to follow the correct polarity when installing the batteries. Reversed batteries may cause damage to the device.
- Do not mix different types of batteries together (e.g. Alkaline and Carbon-zinc or rechargeable batteries) or old batteries with fresh ones. Always replace batteries as a simultaneous set.
- If the batteries in the device are depleted or the device will not be used for a long period of time, remove the batteries to prevent damage or injury from possible battery leakage.
- Do not try to recharge batteries not intended to be recharged; they can overheat and rupture (follow battery manufacturer's directions).
- Do not dispose of batteries in fire, batteries may explode or leak.
- Clean the battery contacts and also those of the device prior to battery installation.
- Remove discharged batteries from the product and dispose/recycle in compliance with all applicable laws.
- Keep batteries away from children and pets. Batteries may be harmful if swallowed. Should a child or pet swallow a battery, seek medical assistance immediately.

#### Changing the weight unit

The scale is set to weigh in pounds (lb) from the factory. Switch the scale on by briefly applying pressure to the top surface with your foot for 1-2 seconds. Wait until "0.0" appears in the display. Press the unit button on the rear of the scale until the desired weight unit appears in the LCD Display ("lb" for pounds, "kg" for kilograms or "st" for stones).

#### Positioning the scale

Place the scale on a hard, flat surface with no carpet or rugs; a hard surface is crucial to accurate measurements.

## 5. Entering User Data

To calculate your body fat percentage and other values, you must save your personal user data. The scale can store ten separate users, which allows you and members of your family to save and access each of your personal settings.

1. With the scale on a hard, flat surface, switch the scale on by briefly applying pressure to the surface with your foot (approx. 1-2 seconds). Wait until "0.0" and the selected measurement unit appears in the LCD Display.
2. Press the SET Button. The first memory space flashes in the display.
3. Press the ▲/▼ button to select the desired memory space and confirm by pressing the SET Button.
4. You can now set the following values:

Memory position	1 to 10
Body size	3'03" to 7'03" (100 cm to 220 cm)
Age	10 to 100 years
Sex	male  / female 
Activity level	A1-A5

### Activity levels

Select an activity level that best reflects your lifestyle.

Activity Level	Physical Activity
1	None
2	Low: Small amount of light physical exertion (e.g. going for walks, easy gardening, simple gymnastics)
3	Medium: Physical exertion at least 2 to 4 times a week for 30 minutes each
4	High: Physical exertion at least 4 to 6 times a week for 30 minutes each
5	Very high: Intense physical exertion, training or hard physical work at least an hour every day

1. Change values by pressing the ▲ button or the ▼ button, or press and hold to quickly scroll through the options.
2. Confirm the entries by pressing the SET button.
3. Once the values have been set, "0.0 lb" is displayed for 20 seconds.
4. Now stand on the conductors of the scale with bare feet while "0.0 lb" is displayed.
5. After your body weight is displayed, your body fat (BF), water and muscle percentage are displayed.

## 6. Taking Measurements

After all parameters have been entered, you can now measure your weight, body fat and the other data.

- Switch on the scale.
- Select your memory space where you would like to save your basic personal data by pressing the "up" ▲ or "down" ▼ button several times. These are then displayed briefly until "0.0" appears.
- Step onto the scale with bare feet and make sure that you are standing still on the conductors.
- All values determined appear after weighing.

**Important:** There may be no contact between feet, legs, calves and thighs. Otherwise the measurement will not be performed correctly.

## Weight Measurement Only

Step on the scale. Stand still with your weight distributed equally on both feet. The scale begins to take measurements straight away. The weight is displayed.

## 7. Evaluating Results

### Body fat percentage

The following body fat percentages are a guideline only; for further information, ask your doctor

#### Men

Age	very good	good	average	poor
10-14	<11 %	11-16 %	16.1-21 %	>21.1 %
15-19	<12 %	12-17 %	17.1-22 %	>22.1 %
20-29	<13 %	13-18 %	18.1-23 %	>23.1 %
30-39	<14 %	14-19 %	19.1-24 %	>24.1 %
40-49	<15 %	15-20 %	20.1-25 %	>25.1 %
50-59	<16 %	16-21 %	21.1-26 %	>26.1 %
60-69	<17 %	17-22 %	22.1-27 %	>27.1 %
70-100	<18 %	18-23 %	23.1-28 %	>28.1 %

#### Women

Age	very good	good	average	poor
10-14	<16 %	16-21 %	21.1-26 %	>26.1 %
15-19	<17 %	17-22 %	22.1-27 %	>27.1 %
20-29	<18 %	18-23 %	23.1-28 %	>28.1 %
30-39	<19 %	19-24 %	24.1-29 %	>29.1 %
40-49	<20 %	20-25 %	25.1-30 %	>30.1 %
50-59	<21 %	21-26 %	26.1-31 %	>31.1 %
60-69	<22 %	22-27 %	27.1-32 %	>32.1 %
70-100	<23 %	23-28 %	28.1-33 %	>33.1 %

Athletes will usually have lower values. Depending on the sport performed, the training intensity and the physical constitution, the results may still be below the stated standard values. Please note, however, that there may be a risk to health in the case of extremely low values.

### Body water content

The body water content in percent is normally within the following ranges:

#### Man

Age	poor	good	very good
10-100	<50	50-65%	>65

#### Woman

Age	poor	good	very good
10-100	<45	45-60%	>60

Body fat contains relatively little water; therefore, body water content may be below the standard values in persons with a high body fat percentage. On the other hand, some athletes may exceed the standard values due to having low levels of fat and a high muscle percentage.

The body water calculation performed using this scale is not suitable for drawing medical conclusions, such as in the case of age-related water retention. Consult your doctor where necessary. A high body water content is generally desirable.

### Muscle percentage

Muscle percentage is normally within the following ranges:

#### Man

Age	low	normal	high
10-14	<44	44-57%	>57%
15-19	<43	43-56%	>56%
20-29	<42	42-54%	>54%
30-39	<41	41-52%	>52%

#### Woman

Age	low	normal	high
10-14	<36	36-43%	>43%
15-19	<35	35-41%	>41%
20-29	<34	34-39%	>39%
30-39	<33	33-38%	>38%

Age	low	normal	high
40–49	<40	40–50%	>50%
50–59	<39	39–48%	>48%
60–69	<38	38–47%	>47%
70–100	<37	37–46%	>46%

Age	low	normal	high
40–49	<31	31–36%	>36%
50–59	<29	29–34%	>34%
60–69	<28	28–33%	>33%
70–100	<27	27–32%	>32%

**NOTE:**

- Only long-term trends are important. Brief deviations in weight within a few days are normally caused by loss of fluids.

Interpretation of these results is based on changes in total body weight, percentage of body fat, body water and muscle content, as well as the length of time over which these changes occur. Rapid changes within the scope of a few days are considered separately from medium-term changes (in the scope of weeks) and long-term changes (months). As a basic rule, short-term changes in weight are almost entirely changes in water content, whereas medium-term and long-term changes may relate to fat percentage and muscle percentage.

If you experience short-term weight loss but your body fat percentage increases or stays the same, you have only lost water, such as after a training session, visit to the sauna, or a diet aimed only at fast weight loss.

On the other hand, if your weight increases in the medium term and your body fat percentage drops or stays the same, you may have built up valuable muscle mass.

If your weight and body fat percentage both fall at the same time, then your diet is working – you are losing fat mass.

Ideally, you should support your diet with physical activity, fitness or strength training. This enables you to increase your muscle percentage in the medium term.

Body fat, body water and muscle percentages must not be added (muscle tissue also contains components made of body water).

## 8. Care and Maintenance

Clean the scale periodically using a damp cloth and, if necessary, a small amount of mild detergent.

## 9. Troubleshooting

If the scale encounters an error during measurement, “FFFF” or “Err” is displayed. If you step onto the scale before it displays “0.0”, incorrect values may display.

Error display	Possible causes of errors	Correction of errors
„FFFF“	The maximum capacity of 400 lb (180 kg, 28 st) has been exceeded.	Weigh only the maximum permissible weight.
„Err“	The electrical resistance between the conductors and the soles of the feet is too high (e.g. in the case of thick calluses or too dry soles).	Repeat the measurement barefoot. If necessary, remove the calluses from the soles of your feet.
„Err“	The fat percentage is outside the measurable range (less than 5 % or greater than 50 %).	Repeat the measurement barefoot. If necessary, remove the calluses from the soles of your feet.
„Err“	The water percentage is outside the measurable range (less than 25 % or greater than 75 %).	Repeat the measurement barefoot. If necessary, remove the calluses from the soles of your feet.
„Err“	The muscle percentage is outside the measurable range (age and gender related).	Repeat the measurement barefoot. If necessary, remove the calluses from the soles of your feet.

Error display	Possible causes of errors	Correction of errors
„Lo“	Low Power.	Change the battery.
	No display.	Battery is empty, battery was inserted incorrectly or the scale is broken.
	Scale shows unrealistic results.	Wait until scale switches off automatically. Repeat measurement. Place the scale on level flooring (no carpet). Don't move during weighing.

## 10. Technical specifications

Battery type	2 x 3 V lithium batteries CR2032
Measuring range	6.6 lb to 400 lb (3 kg to 180 kg)
Switch activation force	11 lb (5 kg)
Weight unit	0.2 lb increments (100 g)
Body fat, water, muscle percentage	unit 0.1 % increments
Electrical current of B.I.A.	max. 5 $\mu$ A / 1 V
Fat percentage range	5 % to 55 %
Water percentage range	25 % to 75 %
Muscle percentage range	25 % to 80 %
Temperature operating range	41 °F to 95 °F (5 °C to 35 °C)
Storage temperature range	-4 °F to 140 °F (-20 °C to 60 °C)
Scale size	12.8" x 12.8" x 0.9" (320 mm x 320 mm x 24 mm)
Net weight of scale	5 lb (2.3 kg)
Subject to technical modifications.	

## 11. FCC Part 15, Subpart B

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

**NOTE:** This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

## 12. Warranty

### Limited Lifetime Warranty For Original Purchaser

Your Beurer body analysis scale, Model BF221, is warranted to be free from defects in materials and workmanship for the life of the product under normal conditions of intended use and service. This warranty extends only to the original retail purchaser and does not extend to retailers or subsequent owners.

We will, at our option, repair or replace the Beurer body analysis scale, Model BF221, without additional charge, for any part or parts covered by these written warranties. No refunds will be given. Repair or replacement is our only responsibility and your only remedy under this written warranty. If replacement parts for defective materials are not available, Beurer reserves the right to make product substitutions in lieu of repair or replacement.

For warranty service contact our customer service department at **1-800-536-0366** or at **info@beurer.com** to provide a description of the problem. If the problem is deemed to be within the scope of the limited lifetime warranty, you will be asked to mail the product at your costs in its original package with proof of purchase, your name, address and phone number. If the problem is not deemed to be within the scope of the limited lifetime warranty, we will provide a quotation for repair respectively replacement and return shipping fee.

This warranty does not cover damage caused by misuse or abuse; accident; the attachment of unauthorized accessory; alteration to the product; improper installation; misapplication; lack of reasonable care with respect to the product; unauthorized repairs or modifications; improper use of electrical/power supply; normal wear; loss of power; dropped product; malfunction or damage of an operating part as a result of failure to comply with instructions for use or to provide manufacturer's recommended maintenance; transit damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Beurer. This warranty is void if the product is ever used in a commercial or business environment. The maximum liability of Beurer under this warranty is limited to the purchase price actually paid by the customer for the product covered by the warranty, as confirmed by proof of purchase, regardless of the amount of any other direct or indirect damage suffered by the customer.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adaptation to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

**THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. ANY IMPLIED WARRANTIES, OBLIGATIONS, OR LIABILITIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTY OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED IN DURATION TO THE DURATION OF THIS APPLICABLE WRITTEN WARRANTY.** Some states do not allow limitations on how long an implied warranty lasts, so the above limitations may not apply to you.

**IN NO EVENT SHALL BEURER BE LIABLE FOR ANY SPECIAL, INCIDENTAL, INDIRECT OR CONSEQUENTIAL DAMAGES FOR BREACH OF THIS OR ANY OTHER WARRANTY, EXPRESS, IMPLIED OR ANY OTHER THEORY OF LIABILITY, WHATSOEVER.** Some states do not allow the exclusion or limitation of special, incidental, or consequential damages, so the above limitation may not apply to you.

Beurer does not authorize anyone, including, but not limited to, retailers, the subsequent consumer purchaser of the product from a retailer or remote purchaser, to obligate Beurer in any way beyond the terms set forth herein.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate in connection with any products or parts thereof which are repaired, replaced, altered, or modified, without the prior explicitly written consent of Beurer.

This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state.

Distributed by  
Beurer North America LP,  
900 N Federal Hwy Ste 300  
Hallandale Beach, FL 33009 USA  
Made in China

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