READ THIS MANUAL COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT

Keep this manual in a safe location for future reference

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CONTENT

1. IMPORTANT SAFETY NOTES	2	6. To Use	5
2. Package Contents	4	7. Care, Maintenance and Disposal	5
3. Parts and Controls.....	4	8. Specifications.....	6
4. Introduction	4	9. Warranty	6
5. Before First Use.....	5		

SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE

If the instruction manual is damaged or if you no longer have the instruction manual in your possession, please contact customer service. Please see warranty for service contact.

Dear Customer,

Thank you for choosing this Beurer product. Our name stands for high-quality, thoroughly tested products for applications in the areas of heating, weight management, blood pressure, body temperature, pulse, gentle therapy, massage, and beauty.

Please read these instructions for use carefully and keep them for later use. Make them accessible to other users and observe the information they contain.

With kind regards,
Your Beurer team

1. IMPORTANT SAFETY NOTES

Signs and symbols

Whenever used, the following signs identify safety and property damage messages and designate a level of hazard or seriousness.

READ AND UNDERSTAND THIS ENTIRE MANUAL INCLUDING THE SAFETY SECTION AND ALL INSTRUCTIONS AND WARNINGS COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT. FOLLOW ALL SAFETY INSTRUCTIONS AND WARNINGS TO AVOID HAZARDOUS SITUATIONS AND TO USE THIS PRODUCT CORRECTLY.

DANGER

Danger indicates a hazardous situation which, if not avoided, will result in death or serious injury.

WARNING

WARNING indicates a hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION

CAUTION indicates a hazardous situation which, if not avoided, may result in minor or moderate injury.

Device in protection class 2.

IPX0

For homeuse only.

No special protection against ingress of water. Keep dry.

Read these instructions carefully before use.

DANGER **Electric shock**

Just like any other electrical appliance, this Foot Spa Massager must be handled with care and attention in order to prevent the risk of electric shock.

For this reason, the device must only be operated as follows:

- Plug the unit only into an electrical outlet as specified on the rating plate.
- Do not use the device if damage to the device itself or any of its accessories is evident.
- Do not connect the unit to an electrical outlet while filling.
- Never connect the Foot Spa to the power supply when your feet are already in the water. If the device is damaged, this can lead to life-threatening electric shock.
- Never use the appliance during a thunderstorm.
- Always switch the Foot Spa off and remove the plug from the electrical outlet before moving it or cleaning it. Always make sure your hands are dry when handling.
- Do not operate under covers, such as blankets or pillows. Excessive heating can occur and cause fire, electric shock, or injury to persons.

WARNING

- Read these instructions carefully before use.
- Ensure the Foot Spa is undamaged before use. If any malfunction should occur during use, remove your feet from the foot wells, shut the unit off immediately and remove the plug from the electrical outlet. This unit contains no user serviceable parts; never attempt to repair the unit yourself.
- The unit is intended only for the purpose described in these instructions. The manufacturer cannot be held responsible for damage caused by improper or careless use.
- Children are not aware of the dangers associated with electrical appliances. Make sure that the Foot Spa is not used by children without supervision.
- Children must not play with the device.
- Keep children away from the packaging materials. Risk of suffocation.
- Close supervision is necessary when this appliance is used near children.
- Never immerse the unit in water.
- Do not place the unit in a shower, tub or basin to fill.
- If water leaks from the unit, discontinue use immediately.
- Do not clamp or pinch the power cord.
- Route the power cord so that it does not pose a trip hazard.
- Never operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the appliance to a service center for examination and repair.
- Do not use outdoors.
- Never drop or insert any object into any opening.
- Keep the cord away from heated surfaces.
- Do not carry this device by the supply cord or use the cord as a handle.
- Never operate on a soft surface such as a bed, carpet or couch where the air openings may be blocked. Keep the air openings free of lint, hair, and the like.
- The surface of the unit may be hot; it may cause serious burns. Persons insensitive to heat must exercise extra caution when using.
- Do not use over sensitive skin areas or in the presence of poor circulation.
- Consult your doctor if you have any medical condition which affects the legs or feet, including but not limited to phlebitis, varicose veins, or diabetes.
- This Foot Spa is not intended for use by persons (including children) with restricted physical, sensory or mental capabilities and/or by persons lacking the required understanding of the equipment and how it is used, unless they are supervised by a person who is responsible for their safety or they are given instructions on how to use the Foot Spa.

- The device should never be left unattended when plugged in. Unplug from outlet when not in use, and before putting on or taking off parts.
- Do not use the Foot Spa Massager with animals.
- Do not operate where aerosol (spray) products are being used or where oxygen is being administered.
- Never use the device in the vicinity of petrol or other easily flammable materials.
- Do not use the Foot Spa Massager for longer than 40 minutes (danger of overheating) and allow it to cool for at least 15 minutes before using it again.
- Do not use attachments not recommended by the manufacturer.
- Do not open the Foot Spa. Failure to comply invalidates the warranty. You must not adjust or repair the Foot Spa yourself.

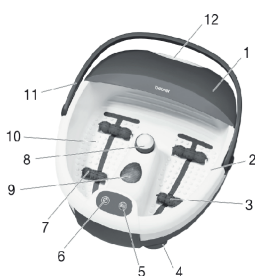
⚠ CAUTION

- Keep the device away from high temperatures.
- The unit is not intended for commercial or industrial use. It is intended solely for household use.
- Do not use this unit longer than 40 minutes continuously. Let unit cool for at least 15 minutes before reuse.
- Do not stand on or in appliance. Use only while seated.
- Empty water only through the drain spout.
- Do not operate the Foot Spa without water except as directed to dry the air passages.
- If you experience pain in a muscle or joint for a prolonged period of time, discontinue use and consult your doctor.
- Never use the Foot Spa on open wounds, discolored areas or any area of the body that is swollen, burned, inflamed or where skin eruption or sores are present.
- Always check the temperature of the water before placing your feet into the Foot Spa.

2. Package Contents

- 1 x FB 21 Foot Spa Massager
- 1 x Operating Instructions
- 3 x Pedicure Attachments with hanger

3. Parts and Controls

1. Splash Guard
2. Max Fill Level Indicator
3. Bubble Strips
4. Rubber Feet (On Bottom)
5. Vibration Button - 
6. Power Button - 
7. Massage Rollers
8. Pedicure Attachment
 - a. Callus Remover
 - b. Brush
 - c. Massage Balls
9. Red Light
10. Foot Wells with Massage Nubs
11. Carrying Handle
12. Drain Spout
13. Cord Storage

4. Introduction

Our feet play a very important role throughout our lives as they constantly bear our full weight. Each foot has 26 bones, 22 joints, and 107 ligaments, and they all need care. Choosing a Beurer Foot Spa is a step in the right direction. The gentle vibration massage relaxes tense muscles and after just a few minutes you will feel more relaxed and stress-free. This effect will be intensified by the spa's aeration effect.

TIPS: Add arnica, rosemary, sea salt extracts, lavender or thyme to warm water to bring relief to tired feet.

5. Before First Use

1. Remove the packaging.
2. Check the device, power supply and cord for damage.
3. Place the foot bath on a firm and level surface.
4. Select a pedicure attachment, place it into the holder between the foot wells, and press firmly.
 - a. The Brush attachment stimulates reflex zones in the soles of your feet.
 - b. The Massage Balls attachment soothes your feet.
 - c. The Callus Remover attachment removes the dead, rough skin of corns and calluses.
5. If not already attached, attach the Splash Guard by raising the Carrying Handle and then pressing the tabs in the Splash Guard into the corresponding slots in the lip of the Foot Spa.

6. To Use

1. Fill the Foot Wells with warm or cold water up to the MAX fill line (1½ in – 2 in, or about 4 - 5 cm). The foot rest should, however, be completely covered with water.

The foot bath is balanced only when it is filled no further than the fill-level mark. Otherwise, the Foot Spa may tip over when the carrying handle is used.

2. Plug the power cord into an electrical outlet. Route the power cord so that it does not pose a trip hazard.
3. Sit comfortably in a chair and place your feet into the Foot Wells. Never stand up in the Foot Spa.
4. The device has two massage functions:
 - Function 1: bubble massage, red light and maintaining of water temperature
 - Function 2: activation of vibration massage

4.1. Bubble massage

To activate the bubble massage (function 1), press the Power Button (ⓘ) once to activate the Foot Spa. Air bubbles will stream out of the Foot Wells and the Red Light will illuminate. These functions can not be switched on or off separately.

Press the Power Button again to shut the bubble massage function off.

4.2. Vibration massage

While the bubble massage with red light and water temperature control (function 1) is running, you can switch on the vibration massage. The vibration massage will only function when function 1 is switched on. It cannot be used separately.

To activate the optional vibration massage (function 2), press the Vibration Button (⊕⊖). Press the Vibration Button again to shut the vibration function off.

5. To experience additional massage effects, move your feet back and forth over the Massage Rollers in the Foot Wells. The rollers can be removed if desired by pulling up on them firmly.
6. Press the Power Button to shut the Foot Spa off.
7. To empty the water, tip the Foot Spa in the direction of the Drain Spout.

7. Care, Maintenance and Disposal

After use, rinse the unit with clean water. The Foot Spa can be cleaned with a mild, non-foaming household cleaner such as neutral vinegar. To dry the air passages, operate without water for about one minute.

Disposal

Observe the local regulations for material disposal. Dispose of the device in accordance with local regulations. If you have any questions, please contact the local authorities responsible for waste disposal.

8. Specifications

Power Supply: 120 VAC, 60 Hz

Electrical Consumption: 60 watts

9. Warranty

Limited Lifetime Warranty For Original Purchaser

Your Beurer Foot Spa Massager, model FB 21, is warranted to be free from defects in materials and workmanship for the life of the product under normal conditions of intended use and service. This warranty extends only to the original retail purchaser and does not extend to retailers or subsequent owners.

We will, at our option, repair or replace the Beurer Foot Spa Massager, model FB 21, without additional charge, for any part or parts covered by these written warranties. No refunds will be given. Repair or replacement is our only responsibility and your only remedy under this written warranty. If replacement parts for defective materials are not available, Beurer reserves the right to make product substitutions in lieu of repair or replacement.

For warranty service contact our customer service department at 1-800-536-0366 or at info@beurer.com to provide a description of the problem. If the problem is deemed to be within the scope of the limited lifetime warranty, you will be asked to mail the product at your costs in its original package with proof of purchase, your name, address and phone number. If the problem is not deemed to be within the scope of the limited lifetime warranty, we will provide a quotation for repair respectively replacement and return shipping fee.

This warranty does not cover damage caused by misuse or abuse; accident; the attachment of unauthorized accessory; alteration to the product; improper installation; misapplication; lack of reasonable care with respect to the product; unauthorized repairs or modifications; improper use of electrical/power supply; old worn batteries; normal wear; loss of power; dropped product; malfunction or damage of an operating part as a result of failure to comply with instructions for use or to provide manufacturer's recommended maintenance; transit damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Beurer. This warranty is void if the product is ever used in a commercial or business environment. The maximum liability of Beurer under this warranty is limited to the purchase price actually paid by the customer for the product covered by the warranty, as confirmed by proof of purchase, regardless of the amount of any other direct or indirect damage suffered by the customer.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adaptation to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. ANY IMPLIED WARRANTIES, OBLIGATIONS, OR LIABILITIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTY OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED IN DURATION TO THE DURATION OF THIS APPLICABLE WRITTEN WARRANTY. Some states do not allow limitations on how long an implied warranty lasts, so the above limitations may not apply to you.

IN NO EVENT SHALL BEURER BE LIABLE FOR ANY SPECIAL, INCIDENTAL, INDIRECT OR CONSEQUENTIAL DAMAGES FOR BREACH OF THIS OR ANY OTHER WARRANTY, EXPRESS, IMPLIED OR ANY OTHER THEORY OF LIABILITY, WHATSOEVER. Some states do not allow the exclusion or limitation of special, incidental, or consequential damages, so the above limitation may not apply to you.

Beurer does not authorize anyone, including, but not limited to, retailers, the subsequent consumer purchaser of the product from a retailer or remote purchaser, to obligate Beurer in any way beyond the terms set forth herein.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate in connection with any products or parts thereof which are repaired, replaced, altered, or modified, without the prior explicitly written consent of Beurer.

This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state.

For more information regarding our product line in the USA, please visit: www.beurer.com

Distributed by:
Beurer North America, LP
900 N Federal Hwy, Ste 300
Hallandale Beach, FL 33009
www.beurer.com

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