**READ THIS MANUAL COMPLETELY AND CAREFULLY BEFORE
USING THIS PRODUCT**

Keep this manual in a safe location for future reference

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Mode d'emploi14

Content

1. IMPORTANT SAFEGUARDS.....	2	6. To Use.....	5
2. Important information.....	4	7. Cleaning and Maintenance.....	5
3. Introduction.....	4	8. Troubleshooting Guide.....	6
4. Package Contents	5	9. FCC Compliance Statement.....	6
5. Parts and Controls.....	5	10. Warranty.....	6

SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE

If the instruction manual is damaged or if you no longer have the instruction manual in your possession, please contact Beurer customer service. Please see warranty for service contact.

Dear Customer,

Thank you for choosing this Beurer product. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air.

Please read these instructions completely and carefully before using this product and keep them for later reference. Be sure to make them accessible to other users and observe the information they contain.

With best regards,
Your Beurer Team

1. IMPORTANT SAFEGUARDS

Signs and symbols

The following signs appear in the Safety Section (page 2) and in this manual on pages 4 and 5.

READ AND UNDERSTAND THIS ENTIRE MANUAL INCLUDING THE SAFETY SECTION AND ALL INSTRUCTIONS AND WARNINGS COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT. FOLLOW ALL SAFETY INSTRUCTIONS AND WARNINGS TO AVOID HAZARDOUS SITUATIONS AND TO MAKE CORRECT USE OF THIS PRODUCT.

WARNING WARNING indicates a hazardous situation which, if not avoided, could result in death or serious injury.

WARNING

- Read these instructions carefully before use.
- Non-observance of the following notes may result in personal injury or material damage (electric shock, skin burns, fire). The following safety and hazard information is not only intended to protect your health and the health of others, it should also protect the product. For this reason, pay attention to these safety notes and include these instructions when handing over the product to others.
- Do not use on small children or people with no heat sensitivity (e.g. diabetics, persons with illness-related skin changes or scarred skin areas in the application area, or after consuming medications or alcohol).
- This foot warmer must not be used by people who are not sensitive to heat or by vulnerable people who may not be able to react to overheating.
- This foot warmer must not be used by children under three.
- This foot warmer is intended for adult use and should not be used by children unless the controller has been set by a parent or responsible adult and the child is closely supervised at all times during use.

- This foot warmer may not be used by persons (including children) with reduced physical, sensory or mental skills or a lack of experience or knowledge, unless they are closely supervised and have been instructed on how to use the foot warmer safely and are fully aware of the risks of use.
- Cleaning and user maintenance must not be performed by children.
- Ensure the device is undamaged before use. If any malfunction should occur during use, remove your feet from the device, shut the device off immediately and remove the plug from the electrical outlet. This device contains no user serviceable parts; never attempt to repair the unit yourself.
- The device is intended only for the purpose described in these instructions. The manufacturer will not be responsible for damage caused by improper or careless use.
- Do not use the device on parts of the body which are inflamed, injured, or swollen. If in doubt, seek medical advice prior to use.
- Excessively long use can cause the skin to be burned.
- Never fall asleep while the device is in use.
- The electric and magnetic fields emitted by this device can interfere with the operation of pacemakers even though electric and magnetic field strengths are far below permissible limits. Consult your doctor and the manufacturer of your pacemaker before using this device.
- The unit is not intended for commercial or industrial use. It is intended solely for household use only.
- Never immerse the device in water.
- Do not switch the device on when it is folded or compressed. Do not pinch the device between objects or bend it sharply.
- Only use in dry surroundings.
- Do not pull on, twist or sharply bend the cables.
- Do not stick pins, needles, or sharp objects into this device.
- Keep this device away from children; it is not a toy.
- Use only the included power supply.
- This foot warmer must not be used to keep animals warm.
- Do not use this foot warmer with shoes.
- Please check this foot warmer frequently for signs of wear and tear or damage. If there are signs indicating the above or if this foot warmer was used improperly, it must be returned to the manufacturer or retailer before further use.
- If the power cord is damaged, it must be replaced by the manufacturer; contact customer service for repair or replacement.
- Always observe the notes on operation (chapter 6) and cleaning and maintenance (chapter 7).
- If you should have any questions about using our devices, please contact Customer Service.

Intended use

This product is intended for warming and massaging human feet.

2. Important information

Signs and symbols

The following symbols are used in these instructions for use, on the packaging and on the type plate for the device and accessories:

	Read instructions!		Do not wash
			Do not bleach
	Do not insert needles!		Do not tumble dry
			Do not iron
	Only use in dry rooms		Do not clean
IPX1	Drip proof		Must not be used by very young children (0-3 years old)
	WARNING; Warning instruction indicating a risk of injury or damage health		
	WARNING; Safety note indicating possible damage to the unit/ accessory		

3. Introduction

Shiatsu is a form of body massage originally developed in Japan around 100 years ago. It is based on ideas anchored in traditional Chinese medicine, in particular the meridian system, i.e. the paths in the human body along which energy is conveyed. The aim of Shiatsu massage is to promote the physical, emotional, and mental well-being of the person being treated by dissolving energy blocks and bottlenecks in the meridians and stimulating the self-regulating ability of the body. Shiatsu therapists achieve this by exerting pressure in flowing movements along the meridians.

Although Shiatsu can be literally translated as “finger pressure,” practitioners also use the base of their hands, elbows, and knees as well as their fingers. In the process, the therapist will use his or her full body weight in order to generate the required pressure.

Massage of a specific point on the sole of the foot may influence the corresponding organ of the body. However, we recommend studying relevant literature and asking your doctor before using point reflex zone massage.

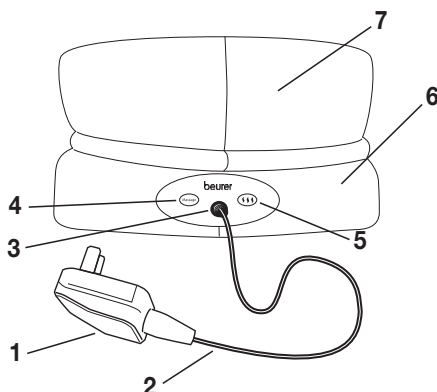
This Shiatsu foot warmer can provide a soothing, effective Shiatsu foot massage with no help required. Besides the intense Shiatsu massage, the foot warmer also provides a warming function. Both functions can be used either at the same time or separately.

4. Package Contents

- 1 x Shiatsu foot warmer with lining
- 1 x Power supply
- 1 x Instruction manual

5. Parts and Controls

- 1. Power Supply
- 2. Cable
- 3. Power Supply Socket
- 4. Massage Button
- 5. 'Warm' Button
- 6. Shiatsu Foot Warmer
- 7. Lining

6. To Use

1. Connect the power cord to the Power Supply Socket and to a standard 110 VAC electrical outlet.
2. To switch on the Shiatsu massage function, press the Massage Button. The massage heads will begin to rotate. Press the Massage Button again to switch off the Shiatsu massage function.
3. To activate the warming function, press the ☺ Button. Press the ☺ Button again to deactivate. When the function is active, a red LED will illuminate above the ☺ Button.
4. The device will shut itself off after 30 minutes of use. To reactivate, press the Massage Button or ☺ Button again. After each use, unplug the power cord from the electrical outlet.
5. After each use, unplug the power cord from the electrical outlet.

NOTES:

- If you press on the massage heads too firmly, the massage function will shut off to protect the device. To reactivate, you must unplug the power cord and then plug it in again.
- If the foot warmer is being used over several hours, we recommend selecting the lowest heat setting on the controller to avoid overheating the feet, which may lead to burns to the skin.
- During the first use, the device may emit a plastic smell that will soon stop.

7. Cleaning and Maintenance

WARNING

Before cleaning, always remove the power supply from the socket.

Do not wash the device or immerse it in water or other liquids. To spot clean, use a damp cloth or sponge and, if necessary, a small amount of liquid detergent. Do not place the device in a clothes dryer. The foot warmer may not be spun, dry-cleaned, wrung out, pressed or ironed. The removable lining is machine washable at 86°F (30°C). Do not use the foot warmer until it is completely dry.

For long-term storage, place the device in its original packaging and store it in a dry location, uncovered. Let the device cool completely before storing.

8. Troubleshooting Guide

Problem	Cause	Solution
The foot warmer does not warm up or the massage function cannot be activated while: – the power supply is connected to a working socket – the power supply cable is connected to the Power Supply Socket on the foot warmer – both functions (heat + massage) are switched on	The foot warmer is faulty.	Contact Customer Service.

9. FCC Compliance Statement

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

10. Warranty

Limited Lifetime Warranty For Original Purchaser

Your Beurer Shiatsu foot warmer, Model FWM50 is warranted to be free from defects in materials and workmanship for the life of the product under normal conditions of intended use and service.

We will, at our option, repair or replace the Beurer Shiatsu foot warmer, Model FWM50, without additional charge, for any part or parts covered by these written warranties. No refunds will be given. Repair or replacement is our only responsibility and your only remedy under this written warranty. If replacement parts for defective materials are not available, the warrantor reserves the right to make product substitutions in lieu of repair or replacement.

For warranty service contact our **customer service department at 1-800-536-0366** to provide a description of the problem. If the problem is deemed to be within the scope of the limited lifetime warranty, you will be asked to mail the product at your costs in its original package with proof of purchase, your name, address and phone number. If the problem is not deemed to be within the scope of the limited lifetime warranty, we will provide a quotation for repair, respectively, replacement and return shipping fees.

This warranty does not cover damage caused by misuse or abuse; accident; the attachment of unauthorized accessories; alteration to the product; improper installation; misapplication; lack of reasonable care with respect to the product; unauthorized repairs or modifications; improper use of electrical/power supply; old worn batteries; normal wear; loss of power; dropped product; malfunction or damage of an operating part as a result of failure to comply with instructions for use or to provide manufacturer's recommended maintenance; transit

damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of the warrantor. This warranty is void if the product is ever used in a commercial or business environment. The maximum liability of the warrantor under this warranty is limited to the purchase price actually paid by the customer for the product covered by the warranty, as confirmed by proof of purchase, regardless of the amount of any other direct or indirect damage suffered by the customer.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adoption to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. ANY IMPLIED WARRANTIES, OBLIGATIONS, OR LIABILITIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTY OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED IN DURATION TO THE DURATION OF THIS APPLICABLE WRITTEN WARRANTY. Some states do not allow limitations on how long an implied warranty lasts, so the above limitations may not apply to you.

IN NO EVENT SHALL THE WARRANTOR BE LIABLE FOR ANY SPECIAL, INCIDENTAL, INDIRECT OR CONSEQUENTIAL DAMAGES FOR BREACH OF THIS OR ANY OTHER WARRANTY, EXPRESS, IMPLIED OR ANY OTHER THEORY OF LIABILITY, WHATSOEVER.

Some states do not allow the exclusion or limitation of special, incidental, or consequential damages, so the above limitation may not apply to you.

The warrantor does not authorize anyone, including, but not limited to, retailers, the subsequent consumer purchaser of the product from a retailer or remote purchaser, to obligate the warrantor in any way beyond the terms set forth herein.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate in connection with any products or parts thereof which are repaired, replaced, altered, or modified, without the prior explicitly written consent of the warrantor.

This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state.

Distributed by:
Beurer North America, LP
900 N Federal Hwy, Ste 300
Hallandale Beach, FL 33009 USA
www.beurer.com

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