**READ THIS MANUAL COMPLETELY AND
CAREFULLY BEFORE USING THIS PRODUCT**

Keep this manual in a safe location for future reference

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Manual de instrucciones8

Dear Customer,

Thank you for choosing this Beurer product. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air.

Please read these instructions completely and carefully before using this product and keep them for later reference. Be sure to make them accessible to other users and observe the information they contain.

With best regards,
Your Beurer Team

Content

1. IMPORTANT SAFETY NOTES	2	5. Operation	4
2. Package Contents	3	6. Cleaning, maintenance and disposal	4
3. Interesting facts.....	3	7. FCC Statement	5
3.1 Weighing correctly	3	8. Technical specifications	5
3.2 General notes.....	3	9. Troubleshooting.....	5
4. Batteries.....	3	10. Warranty.....	6

SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE

If the instruction manual is damaged or if you no longer have the instruction manual in your possession, please contact Beurer customer service. Please see warranty for service contact.

1. IMPORTANT SAFETY NOTES

Signs and symbols

The following signs appear in the Safety Section (pages 2, 3) and in this manual on pages 4 and 5.

READ AND UNDERSTAND THIS ENTIRE MANUAL INCLUDING THE SAFETY SECTION AND ALL INSTRUCTIONS AND WARNINGS COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT. FOLLOW ALL SAFETY INSTRUCTIONS AND WARNINGS TO AVOID HAZARDOUS SITUATIONS AND TO MAKE CORRECT USE OF THIS PRODUCT.

WARNING

WARNING indicates a hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION

CAUTION indicates a hazardous situation which, if not avoided, may result in minor or moderate injury.

NOTICE

NOTICE addresses practices not related to personal injury, such as product and/or property damage.

WARNING To reduce the risk of serious personal injury:

- Keep batteries away from children. Batteries may be harmful if swallowed. Should a child or pet swallow a battery, seek medical assistance immediately. Only adults should handle batteries.
- Swallowing batteries and/or battery fluid can be extremely dangerous. Keep the battery and scale out of the reach of children and disabled persons. Should any person swallow a battery and/or battery fluid, please call 911 immediately.
- Should battery fluid leak and come into contact with your eyes or skin, immediately rinse with plenty of clean water. Please call 911 immediately.
- Batteries should not be charged or reactivated by any other means.
- Batteries should not be taken apart, thrown in the fire or short circuited.

CAUTION To prevent personal injury and/or product/property damage:

- Keep scale away from moisture, dust, chemicals, toiletries, liquid cosmetics, extreme hot and cold temperatures, and sources of heat (e.g., open flame, radiators).
- Prevent impact with hard objects.
- This product does not and is not intended to provide a medical diagnosis. For medical advice, contact a licensed physician. If you have an emergency, call 911 immediately.
- Consult your physician or healthcare provider before beginning a weight reduction or exercise program.
- Do not step on the edge of the scale; otherwise it may tip.
- Do not jump on the scale.
- Strong electromagnetic fields (e.g. cell phones) may impair the accuracy of the scale.
- The scale is not intended to be used for commercial purposes.
- Repairs may only be performed by an authorized service center. Contact Customer Service with any questions regarding operation, usage, or repair of this product.
- Packaging materials are a deadly hazard for children and can cause suffocation. Remove all packaging materials immediately and keep them away from children at all times.
- Do not drop the scale or drop any objects on it.
- Do not use the unit on slippery surfaces such as wet floors. There is a danger of slipping.
- Do not step on the scale if you weigh more than 400 lb (181.4 kg/28.57 st).
- Do not use this product for any purpose other than weighing human beings.
- Place the scale on level flooring, otherwise it may tip.

2. Package Contents

- GS21 Glass Scale
- 1 x 3 V Lithium battery CR2032
- this instruction manual

3. Interesting facts

3.1 Weighing correctly

- Place the scale on a hard, plane level floor (no carpet); a firm floor covering is required for correct measurement.
- If possible, always weigh yourself at the same time of day (preferably in the morning) striving for comparable conditions, on an empty stomach and unclothed, in order to achieve comparable results.
- Stand still and upright during measurement.

3.2 General notes

- The capacity of the scale is 400 lb (181.4 kg/28.57 st).
- The results are shown in 0.2 lb increments (100 g/0.02 st).
- On delivery the scale is set to the unit “lb”. A switch located on the back of the scale enables you to change to “kg” or “st”.

Measuring units

lb	pound
kg	kilogram: the base unit of mass in the Metric System.
st	stone: only for countries where British English is spoken.

For product information, questions or customer service call toll free 1-800-536-0366 or email info@beurer.com


4. Batteries

BATTERY SAFETY PRECAUTIONS

1. Use only the size and type of batteries specified.
2. Be sure to follow the correct polarity when installing the batteries. Reversed batteries may cause damage to the device.

3. If the batteries in the device are depleted or the device will not be used for a long period of time, remove the batteries to prevent damage or injury from possible battery leakage.
4. Do not try to recharge batteries not intended to be recharged; they can overheat and rupture (follow battery manufacturer's directions.)
5. Do not dispose of batteries in fire, batteries may explode or leak.
6. Clean the battery contacts and also those of the device prior to battery installation.
7. Remove discharged batteries from the product and dispose/recycle in compliance with all applicable laws.

If present, remove the battery insulating strip or remove the battery's protective film and insert the battery according to the polarity. If the scale fails to operate, remove the battery completely and reinsert it. Replace the battery cover carefully.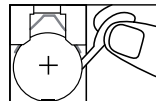

A new battery will last for approximately 2 years if the scale is used three times per day. The trial battery supplied with the unit may have a shorter life. Remove battery if you do not intend to use the scale for longer periods before placing the device in storage.

Your scale is equipped with a "replace battery" indicator. When the battery is weak, "Lo" appears in the display field and the scale automatically switches off. The battery must be replaced (1 x 3 V lithium battery CR2032).

5. Operation


This scale is used to determine the weight of human beings and has a maximum weight of 400 lb (181.4 kg/28.57 st).

Place the scale on a hard, plane level floor (no carpet); a firm floor covering is required for correct measurement.

Note: For first use and relocation

In order to ensure accurate results it is recommended to reset the scale to "0".


- Step on to the scale for one second and then step off.
- The scale shows "0.0 lb" (kg/st).
- It will shut off automatically after several seconds.

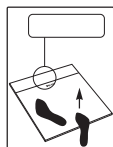Quickstart: Step on to the scale.

Stand on the scale without moving and distribute your weight evenly on both feet.

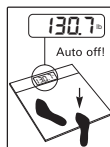After the weight is displayed step off the scale. The scale will automatically turn off after several seconds.

6. Cleaning, maintenance and disposal

NOTICE:

- Protect the scale against hard knocks, moisture, dust, chemicals, major temperature fluctuations and heat sources which are too close (e.g. stoves, heating radiators). This may damage the unit.
- Do not use any aggressive solvents, cleaning agents, detergents or any other strong chemicals to clean the device.
- Never immerse and or spill water or any other liquid onto the scale, otherwise liquid will enter the scale and cause damage.
- Observe the local regulations for material disposal. Dispose of the device in accordance with local regulations.
- Never attempt to repair the instrument or adjust it yourself. This may damage the unit and impair the functions. If you need to have the unit repaired, please contact our customer service. Please see warranty for service contact. Before submitting any complaint, first check the battery and replace it if necessary.
- Changes or modifications to the device will nullify the user warranty.

Cleaning and maintenance

To keep your scale in the best condition and protect the unit from damage:

Clean scale occasionally with a damp cloth. Do not use aggressive cleaning agents. The unit is not waterproof; do not immerse the unit in water and do not spill water on the unit. If your scale gets wet immediately clean off the water. Store the unit in a safe and dry location. Do not expose the unit to direct sunlight, extreme hot or cold temperatures or humidity. Thoroughly dry any moisture off the unit before storing. Store the unit safely out of the reach of children. Do not subject the unit to strong shocks, such as dropping the unit on the floor. Always use the unit following the instructions provided in this manual.

Disposal

NOTICE:

- Always dispose of batteries as per your local regulations.
- Observe the local regulations for material disposal. Dispose of the device in accordance with local regulations. If you have any questions, please contact the local authorities responsible for waste disposal.

7. FCC Statement

Changes or modifications to the product not expressly approved by the manufacturer could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules.

These limits are designed to provide reasonable protection against harmful interference in a residential installation. The equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and the receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

8. Technical specifications

Battery type	1 x 3 V lithium battery CR2032
Measuring range	11 lb to 400 lb (5.0 kg to 181.4 kg/0.79 st to 28.57 st)
Weight increment	0.2 lb (100 g/0.02 st)
Temperature operating range	41 °F to 95 °F (5 °C to 35 °C)
Storage temperature range	-4 °F to 140 °F (-20 °C to 60 °C)
Scale size	12.6" x 12.6" x 0.8" (320 mm x 320 mm x 21mm)
Net weight of scale	4 lb (1.8 kg)
Subject to technical modifications.	

9. Troubleshooting

Following error messages can appear.

Error display	Possible causes	Solution
oL	The maximum capacity of 400 lb (181.4 kg/28.57 st) has been exceeded.	Do not exceed maximum weight.
L0	Low Power.	Change the battery.
	No display.	Battery is depleted, battery was inserted incorrectly or the scale is broken.

	Scale shows unrealistic result.	Wait until scale switches off automatically. Repeat measurement. Place the scale on level flooring (no carpet). Don't move during weighing.
--	---------------------------------	--

10. Warranty

Limited Lifetime Warranty For Original Purchaser

Your Beurer Glass Scale, Model GS21, excluding the battery, is warranted to be free from defects in materials and workmanship for the life of the product under normal conditions of intended use and service. This warranty extends only to the original retail purchaser and does not extend to retailers or subsequent owners. We will, at our option, repair or replace the Beurer Glass Scale, Model GS21, without additional charge, for any part or parts covered by these written warranties. No refunds will be given. Repair or replacement is our only responsibility and your only remedy under this written warranty. If replacement parts for defective materials are not available, Beurer reserves the right to make product substitutions in lieu of repair or replacement.

For warranty service contact our customer service department at **1-800-536-0366** or at **info@beurer.com** to provide a description of the problem. If the problem is deemed to be within the scope of the limited lifetime warranty, you will be asked to mail the product at your costs in its original package with proof of purchase, your name, address and phone number. If the problem is not deemed to be within the scope of the limited lifetime warranty, we will provide a quotation for repair respectively replacement and return shipping fee.

This warranty does not cover damage caused by misuse or abuse; accident; the attachment of unauthorized accessory; alteration to the product; improper installation; misapplication; lack of reasonable care with respect to the product; unauthorized repairs or modifications; improper use of electrical/power supply; old worn batteries; normal wear; loss of power; dropped product; malfunction or damage of an operating part as a result of failure to comply with instructions for use or to provide manufacturer's recommended maintenance; transit damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Beurer. This warranty is void if the product is ever used in a commercial or business environment. The maximum liability of Beurer under this warranty is limited to the purchase price actually paid by the customer for the product covered by the warranty, as confirmed by proof of purchase, regardless of the amount of any other direct or indirect damage suffered by the customer.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adaptation to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. ANY IMPLIED WARRANTIES, OBLIGATIONS, OR LIABILITIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTY OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED IN DURATION TO THE DURATION OF THIS APPLICABLE WRITTEN WARRANTY. Some states do not allow limitations on how long an implied warranty lasts, so the above limitations may not apply to you.

IN NO EVENT SHALL BEURER BE LIABLE FOR ANY SPECIAL, INCIDENTAL, INDIRECT OR CONSEQUENTIAL DAMAGES FOR BREACH OF THIS OR ANY OTHER WARRANTY, EXPRESS, IMPLIED OR ANY OTHER THEORY OF LIABILITY, WHATSOEVER. Some states do not allow the exclusion or limitation of special, incidental, or consequential damages, so the above limitation may not apply to you.

Beurer does not authorize anyone, including, but not limited to, retailers, the subsequent consumer purchaser of the product from a retailer or remote purchaser, to obligate Beurer in any way beyond the terms set forth herein.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate in connection with any products or parts thereof which are repaired, replaced, altered, or modified, without the prior explicitly written consent of Beurer.

This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state.

For more information regarding our product line in the USA, please visit: www.beurer.com

Manufactured for:
Beurer North America LP
900 N Federal Hwy, Suite 300
Hallandale Beach, FL 33009. USA
www.beurer.com

Questions or comments? Call toll free 1-800-536-0366 or contact info@beurer.com
Made in China.

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