ENGLISH	Hearing aid	
	Instructions for use	2
ESPAÑOL	Amplificador de sonido	
	Instrucciones de uso	9
FRANÇAIS	Prothèse auditive	
	Mode d'emploi	17

**READ THIS MANUAL COMPLETELY AND CAREFULLY
BEFORE USING THIS PRODUCT**

Keep this manual in a safe location for future reference

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Contents

1. IMPORTANT SAFETY INSTRUCTIONS	2	6. Care and Maintenance	6
2. Important notes	4	7. Troubleshooting	6
3. Battery Handling Safety Precautions	4	8. Specifications	7
4. Parts and Controls	5	9. Warranty	7
5. To Use	5		

SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE

If the instruction manual is damaged or if you no longer have the instruction manual in your possession, please contact Beurer customer service. Please see warranty for service contact.

Dear Customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air. Please read these instructions completely and carefully before using this product and keep them for later reference. Be sure to make them accessible to other users and observe the information they contain.

With kind regards,
Your Beurer team

1. IMPORTANT SAFETY INSTRUCTIONS

Signs and symbols

The following signs appear in the Safety Section (page 2).

READ AND UNDERSTAND THIS ENTIRE MANUAL INCLUDING THE SAFETY SECTION AND ALL INSTRUCTIONS AND WARNINGS COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT. FOLLOW ALL SAFETY INSTRUCTIONS AND WARNINGS TO AVOID HAZARDOUS SITUATIONS AND TO MAKE CORRECT USE OF THIS PRODUCT.

CAUTION

CAUTION indicates a hazardous situation which, if not avoided, may result in minor or moderate injury.

CAUTION

- The device is intended solely for the purpose described in these operating instructions. The manufacturer is not liable for damage occurring as a result of inappropriate or careless use.
- This product is not intended for use by individuals younger than 18 years old.
- In the event of an existing or previous ear problem, you should consult your doctor before using the hearing aid.
- Do NOT use the hearing aid if you are suffering from an ear inflammation or infection.
- Do NOT use the hearing aid if you are suffering from an inflammation of the middle ear or ear canal.


- Do NOT use the hearing aid at maximum volume over a longer period of time. This could damage your hearing.
- If you feel unwell or experience pain when wearing the hearing aid, stop using it immediately and contact your doctor.
- If your ear becomes inflamed or your skin irritated or you experience an increased accumulation of ear wax when wearing the hearing aid, stop using it immediately and contact your doctor.
- Stop using the hearing aid and seek medical attention if the following occurs:
 - Rash or eczema on areas of skin that contact the hearing aid. The parts of the hearing aid that contact the skin are made from a hypoallergenic material. In rare cases, however, allergic reactions may occur that appear in the form of eczema.
 - Use of the hearing aid results in discharge from the ear and/or the ear requires treatment.
 - Sudden, significant decrease in hearing ability.
- If parts of the hearing aid become stuck in the ear, seek medical attention immediately. Do not attempt to remove the parts yourself.
- At first, set the volume to a low setting and increase it gradually once you have become accustomed to the hearing aid.
- Do NOT set the volume control too high, as this may cause damage to hearing over an extended period of time. This is particularly important to note, as the device achieves a high output power.
- Do NOT lend your hearing aid to others or use the hearing aids of others, as there is a risk of hearing damage and infection.
- The hearing aid is NOT suitable for use by children.
- Keep the hearing aid, the accessories and the batteries away from children, as there is a risk of injury and choking.
- This device is NOT intended for use by people with restricted physical, sensory or mental skills or a lack of experience and/or a lack of knowledge, unless they are supervised by a person who has responsibility for their safety or they receive instructions from this person on how to use the device. Supervise children around the device to ensure they do not play with it.
- Do NOT use the device in oxygen-rich environments or over an extended period of time.
- Please contact the manufacturer in the event of the following:
 - if you require assistance in the set-up, use or maintenance of the hearing aid; or
 - if the device behaves unexpectedly during operation or otherwise.
- Do NOT use a damaged hearing aid. If you have any doubts, contact Customer Service.
- Switch the device off immediately if it is not working properly.
- Do NOT attempt to open and/or repair the hearing aid yourself; contact Customer Service.
- Do NOT wear the hearing aid in the vicinity of strong electromagnetic fields or X-rays.
- Do NOT wear the hearing aid while bathing, showering, or swimming.
- Protect the device from impacts, humidity, dirt, marked temperature fluctuations, direct sunlight, and high levels of humidity.
- High temperatures may develop in closed vehicles during summer. Never leave your hearing aid inside a vehicle.
- Do NOT drop the device or subject it to any shocks.
- If the hearing aid falls into water, dry it thoroughly before use.
- Never attempt to dry the hearing aid in a microwave, a tumble dryer or with a hair dryer.

- Do NOT use the hearing aid together with a mobile phone, as the radio waves of the mobile phone may cause background noise or a loss of volume.
- Do NOT wear the hearing aid while applying cosmetics (e.g. makeup, hairspray, perfume, or sunscreen).
- Do NOT wear the hearing aid while undergoing shortwave therapy or medical treatments that are subject to strong radio frequency or magnetic fields.
- Take care when operating the switch and controller. Do NOT press too firmly.
- Do NOT insert any sharp objects into the microphone opening.
- Do NOT touch the magnets of the hearing aid as this may damage the device.
- Do NOT touch the inside of the device or poke at the inside of the hearing aid with a needle or similar object.
- Maintain your hearing aid regularly. Make sure there are no foreign objects in between the earplug and headphones, as this could impair the sound.
- Do NOT use the device if it has been dropped or otherwise damaged.
- Switch the device off after every use and before every cleaning.
- Service your hearing aid at regular intervals. Make sure there are no foreign objects in between the earplug and headphones, as this could impair the sound.

2. Important notes

Signs and symbols

The following symbols are used in these instructions for use, on the packaging and on the type plate for the device and the accessories:

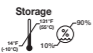	Caution
	Observe the instructions for use
	Application part, type BF
	Permissible operating temperature and humidity.
	Permissible transport and storage temperature. Permissible transport and storage humidity.

3. Battery Handling Safety Precautions

- Use only the size and type of batteries specified.
- Be sure to follow the correct polarity when installing the batteries. Reversed batteries may cause damage to the device.
- Do not mix different types of batteries together (e.g. Alkaline and Carbon-zinc or rechargeable batteries) or old batteries with fresh ones. Always replace batteries as a simultaneous set.
- If the batteries in the device are depleted or the device will not be used for a long period of time, remove the batteries to prevent damage or injury from possible battery leakage.
- Do not try to recharge batteries not intended to be recharged; they can overheat and rupture (follow battery manufacturer's directions.)

- Do not dispose of batteries in fire, batteries may explode or leak.
- Clean the battery contacts and also those of the device prior to battery installation.
- Remove discharged batteries from the product and dispose/recycle in compliance with all applicable laws.
- Keep batteries away from children and pets. Batteries may be harmful if swallowed. Should a child or pet swallow a battery, seek medical assistance immediately.
- If your skin or eyes come into contact with battery fluid, rinse the affected areas with water and seek medical assistance.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Do not use rechargeable batteries.
- Do not disassemble, open or crush the batteries.
- If you are not using the hearing amplifier, switch it off, remove the batteries and store it in the storage box.
- Batteries must NOT be short-circuited.

4. Parts and Controls


- 1 Microphone
- 2 Volume Control
- 3 On/Off Switch "I" = On, "O" = Off
- 4 Battery Compartment
- 5 Earphone
- 6 Earphone Cover
(Small, Medium, or Large)


5. To Use

Remove the hearing aid from the packaging and check that the device and all accessories are included and not damaged.


1. Open the Battery Compartment by pushing down on the tab and swinging the compartment away from the body of the device.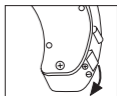

2. Insert one alkaline type LR754 button cell battery (included) into the Battery Compartment observing the correct polarity as indicated at right.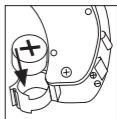

3. Close the Battery Compartment and attach the desired Earphone Cover to the Sound Tube of the Earphone. Make sure the Volume Control is set to the lowest setting before turning on the hearing aid.

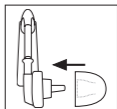4. Bring the hearing aid up to your ear as shown. Gently insert the Earphone into your ear canal, making sure not to insert it too deeply. If the Earphone is not properly in place or feels uncomfortable, try a different size Earphone Cover.

5. Slide the On/Off switch to the on (I) position. Adjust the Volume Control to the desired level so that you can hear sounds clearly and distinctly.

NOTES

- If you hear a whistling sound while using the hearing aid, check for proper earphone fitting. If it is not snug enough, use a different earphone cover.
- The hearing amplifier does not heal an existing hearing impairment, but it does improve hearing and speech understanding.

6. Care and Maintenance

- Remove the battery from the device before cleaning.
- Please always treat the product with care and prevent the Earphone Cover from becoming obstructed with foreign objects. This impairs the function of the sound amplification.
- Clean the hearing aid and the Earphone Cover with a soft damp cloth then wipe dry with a soft, lint-free cloth after each use. If they are very dirty, you can also moisten the cloth with a mild soapy solution. Never use abrasive, corrosive or solvent-based cleaning products.
- Only clean the device as instructed. Always keep liquids from entering the device or accessories.
- Do not use the device until it is completely dry.
- An elongated object or brush works well to remove any ear wax stuck in the earphone cover.
- Allow the hearing aid to dry fully after cleaning. Store the hearing aid in a safe, dry and clean place in the storage box.

7. Troubleshooting

Problem	Possible causes
No sound is heard from the hearing aid.	<ul style="list-style-type: none"> • Hearing aid is not switched on. • Battery is not inserted correctly. • Battery is depleted. • Microphone is dirty. • Hearing aid is damaged.
Sound is very weak.	<ul style="list-style-type: none"> • Volume setting on hearing aid is too low • Battery is almost depleted. • Earphone cover or microphone is dirty. • Earphone may be damaged. • Remove the earphone cover and clean the sound tube and earphone cover. • Battery compartment is not closed. Close the battery compartment completely. • Microphone is dirty.

Problem	Possible causes
The volume control has to be set very high to work properly	<p>Earphone does not fit correctly in the ear, resulting in improper operation or a whistling sound (acoustic feedback). To prevent:</p> <ul style="list-style-type: none"> • Switch off the hearing aid and insert the earphone into the ear correctly. • If the earphone does not fit correctly in the ear, use a different size earphone cover. If you have questions about fit, contact Customer Service.

8. Specifications

Max. output level:	128 dB ± 4 dB
Sound pressure level:	129.4 dB in accordance with IEC 60118-7
Sound amplification:	40 dB ± 5 dB
Harmonic distortion:	≤ 5 % @ 1600Hz
Frequency range:	100–6000Hz
Input noise:	≤ 30 dB
Voltage:	1.5 VDC
Power supply:	≤ 5mA
Operating conditions:	Temperature: 14 °–104 °F (-10 °–40 °C) Relative humidity: 30%–75%
Atmospheric pressure:	20.7–47.25 in Hg (700–1600 hPa)
Storage and transportation conditions:	Temperature: 14 °–131 °F (-10 °–55 °C) Relative humidity: 10%–90%
Atmospheric pressure:	14.76–47.25 in Hg (500–1600 hPa)

9. Warranty

Limited Lifetime Warranty For Original Purchaser

Your Beurer hearing aid, Model HA50, is warranted to be free from defects in materials and workmanship for the life of the product under normal conditions of intended use and service. This warranty extends only to the original retail purchaser and does not extend to retailers or subsequent owners. We will, at our option, repair or replace the Beurer hearing aid, Model HA50, without additional charge, for any part or parts covered by these written warranties. No refunds will be given. Repair or replacement is our only responsibility and your only remedy under this written warranty. If replacement parts for defective materials are not available, Beurer reserves the right to make product substitutions in lieu of repair or replacement.

For warranty service contact our customer service department at **1-800-536-0366** or at **info@beurer.com** to provide a description of the problem. If the problem is deemed to be within the scope of the limited lifetime warranty, you will be asked to mail the product at your costs in its original package with proof of purchase, your name, address and phone number. If the problem is not deemed to be within the scope of the limited lifetime warranty, we will provide a quotation for repair respectively replacement and return shipping fee.

This warranty does not cover damage caused by misuse or abuse; accident; the attachment of unauthorized accessory; alteration to the product; improper installation; misapplication; lack of reasonable care with respect to the product; unauthorized repairs or modifications; improper use of electrical/power supply; normal wear; loss of power; dropped product; malfunction or damage of an operating part as a result of failure to comply with instructions for use or to provide manufacturer's recommended maintenance; transit da-

mage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Beurer. This warranty is void if the product is ever used in a commercial or business environment. The maximum liability of Beurer under this warranty is limited to the purchase price actually paid by the customer for the product covered by the warranty, as confirmed by proof of purchase, regardless of the amount of any other direct or indirect damage suffered by the customer. This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adaptation to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. ANY IMPLIED WARRANTIES, OBLIGATIONS, OR LIABILITIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTY OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED IN DURATION TO THE DURATION OF THIS APPLICABLE WRITTEN WARRANTY. Some states do not allow limitations on how long an implied warranty lasts, so the above limitations may not apply to you.

IN NO EVENT SHALL BEURER BE LIABLE FOR ANY SPECIAL, INCIDENTAL, INDIRECT OR CONSEQUENTIAL DAMAGES FOR BREACH OF THIS OR ANY OTHER WARRANTY, EXPRESS, IMPLIED OR ANY OTHER THEORY OF LIABILITY, WHATSOEVER. Some states do not allow the exclusion or limitation of special, incidental, or consequential damages, so the above limitation may not apply to you.

Beurer does not authorize anyone, including, but not limited to, retailers, the subsequent consumer purchaser of the product from a retailer or remote purchaser, to obligate Beurer in any way beyond the terms set forth herein. This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate in connection with any products or parts thereof which are repaired, replaced, altered, or modified, without the prior explicitly written consent of Beurer.

This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state.

Distributed by:
Beurer North America LP,
900 N Federal Hwy Ste 300
Hallandale Beach, FL 33009 USA
Made in China

Questions or comments?
Call toll-free 1-800-536-0366 or contact info@beurer.com.

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