**READ THIS MANUAL COMPLETELY AND CAREFULLY  
BEFORE USING THIS PRODUCT**

**Keep this manual in a safe location for future reference**

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## CONTENT

1. IMPORTANT SAFETY INSTRUCTIONS.....	2	6. Cleaning.....	4
2. Introduction .....	2	7. Storage.....	4
3. Parts and Controls.....	4	8. Specifications.....	4
4. To Use .....	4	9. Warranty .....	4
5. Care and Storage.....	4		

## SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE

If the instruction manual is damaged or if you no longer have the instruction manual in your possession, please contact Beurer customer service. Please see warranty for service contact.

### Dear Customer,

Thank you for choosing this Beurer product. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air. Please read these instructions completely and carefully before using this product and keep them for later reference. Be sure to make them accessible to other users and observe the information they contain.

With best regards,  
Your Beurer Team

## 1. IMPORTANT SAFETY INSTRUCTIONS

This device is intended solely for the purpose described in these operating instructions. The manufacturer cannot be held liable for inappropriate or careless use.

Whenever used, the following identify safety and property damage messages and designate a level of hazard seriousness.

This safety alert symbol alerts you to potential personal injury hazards. Obey all safety messages that follow it to help avoid injury.

### WARNING

WARNING indicates a hazardous situation which, if not avoided, could result in death or serious injury.

### CAUTION

CAUTION indicates a hazardous situation which, if not avoided, may result in minor or moderate injury.

### NOTICE

NOTICE addresses practices not related to personal injury, such as product and/or property damage.

### NOTE

NOTE provides important information.

## 2. Introduction

This electric massager with heat allows you to give yourself or other people soothing massages. Massages can be relaxing or stimulating. This massager delivers powerful and intensive back massages and is intended for private use only and not for medical or commercial purposes.

## **WARNING:**

- Do not use this massager:
  - On children.
  - On animals.
  - On persons suffering from a herniated disk or from abnormal changes to the vertebrae.
  - To massage the heart region of persons fitted with a pacemaker. Always consult a doctor before using this massager on a person fitted with a pacemaker.
  - On swollen, burned, inflamed or damaged skin or body parts.
  - During pregnancy.
  - On the face/eyes, larynx, or other particularly sensitive parts of the body.
  - While asleep.
  - After taking medication or consuming alcohol.
- This device is not intended for use by persons (including children) with restricted physical, sensory or mental capabilities and/or by persons lacking an understanding of the equipment and how it is used.
- This device is not a toy; keep away from children.
- Consult your doctor before using the massage device if you suffer from:
  - A serious illness or are recovering from an operation.
  - Common cold accompanied by a fever, varicose veins, thrombosis, phlebitis, jaundice, diabetes, nervous system diseases (e.g. sciatica) or acute inflammations.
- Do not operate the device:
  - With a voltage different than that specified on the device.
  - If it or any of its accessories is damaged.
  - During a thunderstorm.
- In the event of damage or a malfunction, switch the device off immediately and disconnect it from the power supply.
- Never pull on the power cable or on the appliance in order to disconnect the plug.
- Never hold or carry the device by the power cable. Keep the cables away from hot surfaces.
- Never open the device for any reason.
- If the power cord of the device is damaged, have it replaced right away.
- Make sure the massage device, plug and the power cord do not come into contact with water or other fluids.
- Operate the device:
  - Only in dry inside rooms
  - Only with dry hands
  - Never in the bath, in the shower or near a filled basin
  - Never in swimming pools, whirlpools or in the sauna
  - Never out in the open
- Never try to retrieve the device if it falls into water. Disconnect the plug immediately from the electrical outlet.
- Repairs to electrical appliances must only be carried out by qualified persons.
- Never leave the device unsupervised, particularly if children are around.
- Never use the device under covers, such as blankets or pillows.

## **CAUTION:**

- Switch the device off and disconnect the power supply after every use and before every cleaning.
- Make sure that all moving parts can move freely at all times.
- Make sure hair cannot get caught in the device.
- Avoid contact with sharp or pointed objects.
- Do not kink the power cord.
- Make sure the device is not too close to heat sources such as ovens, radiators or direct sunlight.:

### 3. Parts and Controls

- 1 - Power Switch  
O: Off  
M: Vibration massage  
M+H: Vibration massage with heat
- 2 - Massage head with heat
- 3 - Massage Attachments

### 4. To Use

Attach one of the supplied Massage Attachments over the massage head. Make sure the device is switched off and then plug the power cord into a standard electrical outlet. Switch the device on by sliding the power switch to the desired massage setting “M” or “M+H.”

Guide the device with both hands to the area of the body to be massaged. You can stand, sit, or lie down. You can alter the massage intensity by pressing down firmly or gently. Do not massage one part of the body longer than three minutes before moving to other areas.

#### **WARNING:**

- Stop the massage or change the position of the massager if it is painful or unpleasant.
- Do not jam or force any part of your body, particularly your fingers, into the massage head.

#### **NOTE**

Do not use the massage device right before going to bed. The massage can have a stimulating effect and may make it difficult to fall asleep.

### 5. Care and Storage

#### **WARNING:**

- Disconnect the device before cleaning it.
- Only clean the appliance in the manner specified. Ensure no fluids enter the device or accessories.
- Do not use the device until it is completely dry.

### 6. Cleaning

Unplug the device and allow it to cool before cleaning it with a damp cloth and, if necessary, a mild detergent.

### 7. Storage

Keep the massage device in a cool and dry place where it is out of the reach of children, preferably in its original packaging.

### 8. Specifications

Dimensions:	2.5" W x 3.1" H x 13" L (65 x 80 x 330 mm)
Weight:	0.9 lbs (0.41 kg)
Rated output:	5W
Rated voltage:	120V ~ 60Hz

### 9. Warranty

#### **Limited Lifetime Warranty For Original Purchaser**

Your Beurer massager with heat, Model MG 21, is warranted to be free from defects in materials and workmanship for the life of the product under normal conditions of intended use and service. This warranty extends only to the original retail purchaser and does not extend to retailers or subsequent owners.

We will, at our option, repair or replace the Beurer massager with heat, Model MG 21, without additional charge, for any part or parts covered by these written warranties. No refunds will be given. Repair or replacement is our only responsibility and your only remedy under this written warranty. If replacement parts for defective materials are not available, Beurer reserves the right to make product substitutions in lieu of repair or replacement.

For warranty service contact our customer service department at **1-800-536-0366** or at **info@beurer.com** to provide a description of the problem. If the problem is deemed to be within the scope of the limited lifetime warranty, you will be asked to mail the product at your costs in its original package with proof of purchase, your name, address and phone number. If the problem is not deemed to be within the scope of the limited lifetime warranty, we will provide a quotation for repair respectively replacement and return shipping fee.

This warranty does not cover damage caused by misuse or abuse; accident; the attachment of unauthorized accessory; alteration to the product; improper installation; misapplication; lack of reasonable care with respect to the product; unauthorized repairs or modifications; improper use of electrical/power supply; normal wear; loss of power; dropped product; malfunction or damage of an operating part as a result of failure to comply with instructions for use or to provide manufacturer's recommended maintenance; transit damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Beurer. This warranty is void if the product is ever used in a commercial or business environment. The maximum liability of Beurer under this warranty is limited to the purchase price actually paid by the customer for the product covered by the warranty, as confirmed by proof of purchase, regardless of the amount of any other direct or indirect damage suffered by the customer.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adaptation to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

**THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. ANY IMPLIED WARRANTIES, OBLIGATIONS, OR LIABILITIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTY OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED IN DURATION TO THE DURATION OF THIS APPLICABLE WRITTEN WARRANTY.** Some states do not allow limitations on how long an implied warranty lasts, so the above limitations may not apply to you.

**IN NO EVENT SHALL BEURER BE LIABLE FOR ANY SPECIAL, INCIDENTAL, INDIRECT OR CONSEQUENTIAL DAMAGES FOR BREACH OF THIS OR ANY OTHER WARRANTY, EXPRESS, IMPLIED OR ANY OTHER THEORY OF LIABILITY, WHATSOEVER.** Some states do not allow the exclusion or limitation of special, incidental, or consequential damages, so the above limitation may not apply to you.

Beurer does not authorize anyone, including, but not limited to, retailers, the subsequent consumer purchaser of the product from a retailer or remote purchaser, to obligate Beurer in any way beyond the terms set forth herein.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate in connection with any products or parts thereof which are repaired, replaced, altered, or modified, without the prior explicitly written consent of Beurer.

This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state. For more information regarding our product line in the USA, please visit: [www.beurer.com](http://www.beurer.com)

Manufactured for:  
Beurer North America LP  
900 N Federal Hwy, Suite 300  
Hallandale Beach, FL 33009. USA  
[www.beurer.com](http://www.beurer.com)

**Questions or comments? Call toll free 1-800-536-0366 or contact [info@beurer.com](mailto:info@beurer.com)**

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