**READ THIS MANUAL COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT**

Keep this manual in a safe location for future reference

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# ENGLISH

## Content

1. IMPORTANT SAFETY INSTRUCTIONS.....	2	7. Care, Maintenance and Disposal.....	5
2. Package Contents.....	4	8. Troubleshooting Guide.....	5
3. Introduction.....	4	9. Specifications.....	5
4. Parts and Controls.....	4	10. Power Adapter.....	6
5. To Set Up.....	4	11. FCC Compliance Statement.....	6
6. To Use.....	4	12. Warranty.....	6

## SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE

If the instruction manual is damaged or if you no longer have the instruction manual in your possession, please contact Beurer customer service. Please see warranty for service contact.

### Dear Customer,

Thank you for choosing this Beurer product. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air.

Please read these instructions completely and carefully before using this product and keep them for later reference. Be sure to make them accessible to other users and observe the information they contain.

With best regards,  
Your Beurer Team

## 1. IMPORTANT SAFETY INSTRUCTIONS

### Signs and symbols

Whenever used, the following signs identify safety and property damage messages and designate a level of hazard or seriousness.

**READ AND UNDERSTAND THIS ENTIRE MANUAL INCLUDING THE SAFETY SECTION AND ALL INSTRUCTIONS AND WARNINGS COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT. FOLLOW ALL SAFETY INSTRUCTIONS AND WARNINGS TO AVOID HAZARDOUS**

	<b>WARNING</b>	WARNING indicates a hazardous situation which, if not avoided, could result in death or serious injury.
	<b>CAUTION</b>	CAUTION indicates a hazardous situation which, if not avoided, may result in minor or moderate injury.
	<b>IMPORTANT</b>	Safety note regarding potential for damage to the device/accessories
	<b>NOTICE</b>	NOTICE addresses practices not related to personal injury, such as product and/or property damage.
		Device in protection class 2
		For indoor use only

## SITUATIONS AND TO MAKE CORRECT USE OF THIS PRODUCT.

	Polarity of the DC voltage connection
	Energy Efficiency class VI

### WARNING:

- Before use, ensure that all packaging materials are removed and that there is no visible damage to the unit or accessories. When in doubt, do not use the unit and contact customer service.
- Always place the daylight lamp on a flat, level, stable, hard surface.
- Do not immerse the unit into water and do not use it in wet rooms such as a bathroom, laundry room, or pool area.
- This product is not intended for commercial use. It is designed exclusively for personal use at home.
- Do not use after taking painkillers or drugs or drinking alcohol.
- Keep children away from packaging materials (risk of suffocation).
- Do not cover or store the unit while it is warm.
- Children must not play with this device.
- The lamp remains hot after use. Let the lamp cool before storing.
- Route the power cord so that it does not pose a trip hazard.
- Always unplug the power adapter and allow the device to cool down before touching it.
- Do not touch the unit with wet hands while it is plugged in; do not allow any water to be sprayed onto the unit. Operate only when the unit is completely dry.
- Ensure that you only insert and remove the power adapter with dry hands and that you only press the ON/OFF button with dry hands.
- Keep the power cord away from heated objects and flames.
- Protect the unit from strong impacts.
- Do not disconnect the power adapter from the electrical outlet by pulling on the cord.
- Do not use the device in the presence of flammable anesthetic gas connections with air, oxygen or nitrogen oxide.
- This product has no user-serviceable parts; return the product to customer service for any repair or replacement.
- If you are taking medication such as pain relievers, anti-hypertensives or antidepressants, consult your doctor before using the daylight lamp.
- People with retinal diseases as well as diabetics need to be examined by an ophthalmologist before using the daylight lamp.
- Do not use if you have an eye disease such as cataracts, glaucoma, diseases of the optic nerve in general and inflammation of the vitreous humor.
- If you have health concerns of any kind, contact your doctor before using this product.
- The light of the daylight lamp is only intended for use on the human body.
- Connect the unit only to the mains voltage listed on the type plate.
- Do not use it in the vicinity of people who are helpless, children under age 3 or people who are insensitive to heat (people with skin lesions due to illness).
- Do not use the device if it shows signs of damage or does not function properly. In these cases, contact Customer Service.
- If the mains connection cable of this device is damaged, it must be disposed of. If it cannot be removed, the device must be disposed of.

- Disconnection from the power supply network is only guaranteed when the mains part is unplugged.
- Do not position the device so that it is difficult to unplug the power adapter.
- Disposal of the device shall be according to local regulations.
- Risk of strangulation resulting from baby or child entanglement in power adapter output cables.
- No calibration and no preventative checks or maintenance is to be carried out on this device.
- You cannot repair the device. The device contains no parts that you can repair.
- Do not make any changes to the device without the manufacturer's permission.
- If the device has been changed, thorough tests and checks must be carried out to ensure the continued safety of further use of the device.
- Only operate the device with the power adapter provided.

## 2. Package Contents

- 1 x Daylight Lamp
- 1 x Storage Bag
- 1 x AC-Adapter
- 1 x Stand (adjustable)
- 1 x Instruction Manual

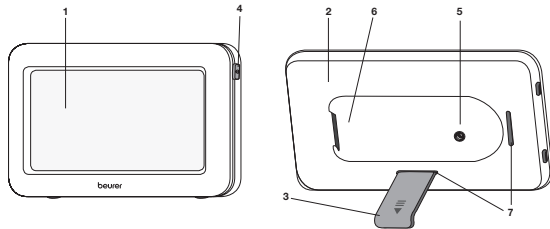
## 3. Introduction

Why use a daylight lamp?

Natural spectrum daylight may help to reduce the effects associated with winter blues, jet lag, shift work and seasonal time changes and boost your sense of well-being.

## 4. Parts and Controls

1. Fluorescent Screen
2. Rear Panel
3. Stand
4. ON/OFF Button
5. Adapter Socket
6. Storage Slot for Stand
7. Holder for Stand

## 5. To Set Up

Remove plastic wrapping and check the device for damage. If you notice any damage, contact customer service at 1-800-536-0366.

1. Place the device on a flat, level, stable, hard surface so that it will be 3.9–11.8 in (10–30 cm) away from you during use.
2. To prevent product damage, only use the supplied power adapter.
3. Insert the round power adapter plug into the Adapter Socket (marked ) on the rear of the Daylight Lamp. Connect the power adapter to a standard 110 VAC electrical outlet.

## 6. To Use

1	Remove the Stand from its storage slot.
2	Clip the Stand into one of the Holders on the housing. The Stand can be attached to either the long (horizontal use) or short side (vertical use). The bend in the stand can be either used facing up or down, which allows positioning the daylight lamp at two different angles.
3	<b>Switching on the lamp</b> ▶ Press the On/Off Button.

4	<p><b>Enjoying the light</b></p> <p>Sit 3.9 – 11.8 in (10 – 30 cm) from the lamp. You can go about your normal activities while using it, such as reading, writing, making telephone calls, etc.</p> <ul style="list-style-type: none"> <li>• Periodically look directly into the light briefly. Do not stare directly at the light for extended periods, since that could lead to overstimulation of the retina.</li> <li>• For more effective results use for at least seven consecutive days.</li> <li>• It is recommended to use the daylight lamp between 6 AM and 8 PM for two hours per day.</li> <li>• Do not leave the daylight lamp unattended while in use.</li> <li>• Start with shorter periods of illumination and increase over the course of a week.</li> </ul> <p><b>NOTE:</b> Eye aches and mild headaches may occur for the first uses and will subside after continued usage.</p>												
5	<p><b>Duration</b></p> <p>The duration of the application depends on the distance. The closer you are to the light source, the shorter the application time:</p> <table border="1"> <thead> <tr> <th>LUX</th> <th>Distance</th> <th>Application duration</th> </tr> </thead> <tbody> <tr> <td>10,000</td> <td>approx. 10 cm</td> <td>0.5 hours</td> </tr> <tr> <td>5,000</td> <td>approx. 20 cm</td> <td>1 hour</td> </tr> <tr> <td>2,500</td> <td>approx. 30 cm</td> <td>2 hours</td> </tr> </tbody> </table>	LUX	Distance	Application duration	10,000	approx. 10 cm	0.5 hours	5,000	approx. 20 cm	1 hour	2,500	approx. 30 cm	2 hours
LUX	Distance	Application duration											
10,000	approx. 10 cm	0.5 hours											
5,000	approx. 20 cm	1 hour											
2,500	approx. 30 cm	2 hours											
6	<p><b>Switching off the lamp</b></p> <p>▶ Press the On/Off button to shut the unit off. Unplug the power supply from the electrical outlet. The LEDs will switch off.</p> <p><b>CAUTION</b> The lamp remains hot after use. Let the lamp cool before storing.</p>												

## 7. Care, Maintenance and Disposal

- Before cleaning the unit, always switch it off, unplug it, and let it cool.
- Do not immerse unit in water or other liquids. Use a slightly damp cloth to clean the device. Do not use abrasive detergents.
- Do not touch the unit with wet hands while it is plugged in; do not allow any water to be sprayed onto the unit. The unit must be operated only when it is completely dry.
- If you are not going to use the device for an extended period of time, disconnect it and store it in a dry place, out of the reach of children.

**Disposal:** Observe the local regulations for material disposal. Dispose of the device in accordance with local regulations. If you have any questions, please contact the local authorities responsible for waste disposal.

## 8. Troubleshooting Guide

Problem	Possible cause	Solution
Device does not light up	On/Off Button switched off.	Switch on the On/Off Button.
	No electricity.	Connect the power cord properly.
	No electricity.	The power cord is faulty. Contact customer service.
	LEDs have reached the end of their service life or are faulty.	Contact customer service.

## 9. Specifications

Type	TL 20
Dimensions (WHD)	approx. 8.3 x 5.1 x 0.8 in (210 x 130 x 20 mm)
Weight	approx. 8.1 oz (230 g); mains part: 2.3 oz (66 g)
Lighting elements	LED
LED color temperature	6,000 (± 300) Kelvin
LED lifetime	10,000 hours
Illuminance	10,000 lux (at a distance of approx. 3.9 in/ 10 cm)
Illumination surface	approx. 21.7 sq in (140 cm <sup>2</sup> )
Power	7.2 Watts
Radiation	Output of radiation beyond the visible spectrum (infrared and UV) is so low that it is harmless to eyes and skin.
Operating conditions	32 °F to 104 °F (0 °C to 40 °C), 30 – 90 % relative humidity, atmospheric pressure: 700 - 1060 hPa
Storage conditions	14 °F to 140 °F (-10 °C to 60 °C), 30 – 90 % relative humidity, atmospheric pressure: 700 - 1060 hPa
Product classification	External power supply, Protection class II

## 10. Power Adapter

Adapter Model	BI12T-120060-AdU
Input	100 – 240 V ~ 50/60 Hz 0.5 A
Output	12 V DC, 600 mA, only in conjunction with TL 20
Protection	The device is double-protected.
Polarity of the DC voltage connection	
Classification	Protection class II

## 11. FCC Compliance Statement

Changes or modifications to the product not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

**NOTE:** This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

## 12. Warranty

### Limited Lifetime Warranty For Original Purchaser

Your Beurer daylight lamp, Model TL 20, excluding the battery, is warranted to be free from defects in materials and workmanship for the life of the product under normal conditions of intended use and service. This warranty extends only to the original retail purchaser and does not extend to retailers or subsequent owners.

We will, at our option, repair or replace the Beurer daylight lamp, Model TL 20, without additional charge, for any part or parts covered by these written warranties. No refunds will be given. Repair or replacement is our only responsibility and your only remedy under this written warranty. If replacement parts for defective materials are not available, Beurer reserves the right to make product substitutions in lieu of repair or replacement.

For warranty service contact our customer service department at **1-800-536-0366** or at **info@beurer.com** to provide a description of the problem. If the problem is deemed to be within the scope of the limited lifetime warranty, you will be asked to mail the product at your costs in its original package with proof of purchase, your name, address and phone number. If the problem is not deemed to be within the scope of the limited lifetime warranty, we will provide a quotation for repair respectively replacement and return shipping fee.

This warranty does not cover damage caused by misuse or abuse; accident; the attachment of unauthorized accessory; alteration to the product; improper installation; misapplication; lack of reasonable care with respect to the product; unauthorized repairs or modifications; improper use of electrical/power supply; old worn batteries; normal wear; loss of power; dropped product; malfunction or damage of an operating part as a result of failure to comply with instructions for use or to provide manufacturer's recommended maintenance; transit damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Beurer. This warranty is void if the product is ever used in a commercial or business environment. The maximum liability of Beurer under this warranty is limited to the purchase price actually paid by the customer for the product covered by the warranty, as confirmed by proof of purchase, regardless of the amount of any other direct or indirect damage suffered by the customer.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adaptation to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

**THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. ANY IMPLIED WARRANTIES, OBLIGATIONS, OR LIABILITIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTY OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED IN DURATION TO THE DURATION OF THIS APPLICABLE WRITTEN WARRANTY.** Some states do not allow limitations on how long an implied warranty lasts, so the above limitations may not apply to you.

**IN NO EVENT SHALL BEURER BE LIABLE FOR ANY SPECIAL, INCIDENTAL, INDIRECT OR CONSEQUENTIAL DAMAGES FOR BREACH OF THIS OR ANY OTHER WARRANTY, EXPRESS, IMPLIED OR ANY OTHER THEORY OF LIABILITY, WHATSOEVER.** Some states do not allow the exclusion or limitation of special, incidental, or consequential damages, so the above limitation may not apply to you.

Beurer does not authorize anyone, including, but not limited to, retailers, the subsequent consumer purchaser of the product from a retailer or remote purchaser, to obligate Beurer in any way beyond the terms set forth herein.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate in connection with any products or parts thereof which are repaired, replaced, altered, or modified, without the prior explicitly written consent of Beurer.

This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state.

For more information regarding our product line in the USA, please visit: [www.beurer.com](http://www.beurer.com)

Distributed by:  
Beurer North America LP  
900 N Federal Highway, Suite 300  
Hallandale Beach, FL 33009  
[www.beurer.com](http://www.beurer.com)

**Questions or comments? Call toll free 1-800-536-0366 or contact info@beurer.com**

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