**READ THIS MANUAL COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT**

Keep this manual in a safe location for future reference

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## SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE

**If the instruction manual is damaged or if you no longer have the instruction manual in your possession, please contact Beurer customer service. Please see warranty for service contact.**

### Dear Customer,

Thank you for choosing this Beurer product. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air.

Please read these instructions completely and carefully before using this product and keep them for later reference. Be sure to make them accessible to other users and observe the information they contain.

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
With best regards,  
Your Beurer Team

## 1. IMPORTANT SAFETY INSTRUCTIONS

### Signs and symbols

The following signs appear in the Safety Section (page 2) and in this manual on page 3 and 4.

**READ AND UNDERSTAND THIS ENTIRE MANUAL INCLUDING THE SAFETY SECTION AND ALL INSTRUCTIONS AND WARNINGS COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT. FOLLOW ALL SAFETY INSTRUCTIONS AND WARNINGS TO AVOID HAZARDOUS SITUATIONS AND TO MAKE CORRECT USE OF THIS PRODUCT.**

	<b>WARNING</b>	WARNING indicates a hazardous situation which, if not avoided, could result in death or serious injury.
	<b>CAUTION</b>	CAUTION indicates a hazardous situation which, if not avoided, may result in minor or moderate injury.
	<b>IMPORTANT</b>	Safety note regarding potential for damage to the device/accessories
	<b>NOTICE</b>	NOTICE addresses practices not related to personal injury, such as product and/or property damage.
		Device in protection class 2
	IP 21	Protected against foreign objects equal to or greater than 12.5 mm in diameter and against vertically falling drops of water
		For home use only

Follow instructions for use

This product satisfies the requirements of the applicable European and national directives.

Serial number

Date of manufacture

## WARNING:

- Before use, ensure that all packaging materials are removed and that there is no visible damage to the unit or accessories. When in doubt, do not use the unit and contact customer service.
- Always place the daylight lamp on a flat, level, stable, hard surface.
- Do not immerse the unit into water and do not use it in wet rooms such as a bathroom, laundry room, or pool area.
- Do not use after taking painkillers or drugs or drinking alcohol.
- Keep children away from packaging materials (risk of suffocation).
- Do not cover or store the unit while it is warm.
- Always unplug the power adapter and allow the device to cool down before touching it.
- Do not touch the unit with wet hands while it is plugged in; do not allow any water to be sprayed onto the unit. Operate only when the unit is completely dry.
- Ensure that you only insert and remove the power adapter with dry hands and that you only press the ON/OFF button with dry hands.
- Keep the power cord away from heated objects and flames.
- Protect the unit from strong impacts.
- Do not disconnect the power adapter from the electrical outlet by pulling on the cord.
- Do not use the device in the presence of flammable anesthetic gas connections with air, oxygen or nitrogen oxide.
- This product has no user-serviceable parts; return the product to customer service for any repair or replacement.
- If you are taking medication such as pain relievers, anti-hypertensives or antidepressants, consult your doctor before using the daylight lamp.
- People with retinal diseases as well as diabetics need to be examined by an ophthalmologist before using the daylight lamp.
- Do not use if you have an eye disease such as cataracts, glaucoma, diseases of the optic nerve in general and inflammation of the vitreous humor.
- If you have health concerns of any kind, contact your doctor before using this product.
- The light of the daylight lamp is only intended for use on the human body.
- Connect the unit only to the mains voltage listed on the type plate.
- Do not use it in the vicinity of people who are helpless, children under age 3 or people who are insensitive to heat (people with skin lesions due to illness).
- Do not use the device if it shows signs of damage or does not function properly. In these cases, contact Customer Service.
- If the mains connection cable of this device is damaged, it must be disposed of. If it cannot be removed, the device must be disposed of.
- Disconnection from the power supply network is only guaranteed when the mains part is unplugged.
- Do not position the device so that it is difficult to unplug the power adapter.
- Disposal of the device shall be according to local regulations.
- Strangulation resulting from baby or child entanglement in power adapter output cables.
- No calibration and no preventative checks or maintenance is to be carried out on this device.

- You cannot repair the device. The device contains no parts that you can repair.
- Do not make any changes to the device without the manufacturer's permission.
- If the device has been changed, thorough tests and checks must be carried out to ensure the continued safety of further use of the device.
- Only operate the device with the power adapter provided.

## 2. Package contents

- 1 x Daylight Lamp
- 1 x AC-Adapter
- 1 x Storage Bag
- 1 x Stand (adjustable)
- 1 x Instruction Manual

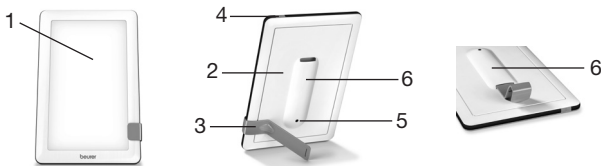
## 3. Introduction

Why use a daylight lamp?

Natural spectrum daylight may help to reduce the effects associated with Winter Blues, jet lag, shift work and seasonal time changes and boost your sense of well-being.

## 4. Parts

1. Fluorescent Screen
2. Rear Panel
3. Stand
4. ON/OFF Button
5. Adapter Socket
6. Storage Slot for Stand

## 5. To Set Up

Remove plastic wrapping and check the device for damage. If you notice any damage, contact customer service at 1-800-536-0366.

1. Place the device on a flat, level, stable, hard surface so that it will be 3.9–7.8 in (10–20 cm) away from you during use.
2. To prevent product damage, only use the supplied power adapter.
3. Insert the round power adapter plug into the Adapter Socket (marked ) on the rear of the Daylight Lamp. Connect the power adapter to a standard 110 VAC electrical outlet.

## 6. To Use

1. Remove the Stand from its storage slot.
2. Clip on the Stand to the side of the housing. The orientation/angle of the Daylight Lamp changes depending on where you attach the Stand, which enables you to tailor the angle as desired. The Stand can be attached to either the long or short side.
3. Press the On/Off Button.
4. Sit 3.9–7.8 in (10–20 cm) from the lamp. You can go about your normal activities while using it, such as reading, writing, making telephone calls, etc.

Usage duration depends on the distance; the closer you are to the light, the less time you should use it.

Distance	Duration
3.9 in (10 cm)	½ hour per day
5.9 in (15 cm)	1 hour per day
7.8 in (20 cm)	2 hours per day

5. Periodically look directly into the light briefly. Do not stare directly at the light for extended periods, since that could lead to overstimulation of the retina.

6. For more effective results use for at least seven consecutive days. Between 6 AM and 8 PM for two hours per day.
7. Start with shorter periods of illumination and increase over the course of a week.

**NOTE:** Eye aches and mild headaches may occur for the first uses and will subside after continued usage.

8. Press the On/Off Button to shut the unit off. Unplug the power adapter from the electrical outlet.

**CAUTION**

The lamp remains hot after use. Let the lamp cool before storing.

## 7. Care and Storage

- Before cleaning the unit, always switch it off, unplug it, and let it cool.
- Do not immerse unit in water or other liquids. Use a slightly damp cloth to clean the device. Do not use abrasive detergents.
- Do not touch the unit with wet hands while it is plugged in; do not allow any water to be sprayed onto the unit. The unit must be operated only when it is completely dry.
- If you are not going to use the device for an extended period of time, disconnect it and store it in a dry place, out of the reach of children.

## 8. Troubleshooting Guide

Problem	Possible cause	Solution
Device does not light up	On/Off Button switched off.	Switch on the On/Off Button.
	No electricity.	Connect the power cord properly.
	No electricity.	The power cord is faulty. Contact customer service.
	LEDs have reached the end of their service life or are faulty.	Contact customer service.

## 9. Specifications

Model	TL 30
Type	WL11
Dimensions (WHD)	approx. 9.3 in x 6.1 in x 1.0 in (236 x 156 x 26 mm)
Weight	approx. 11.6 oz (330 g) (14.3 oz (405 g) incl. mains adapter + storage pouch)
Lighting elements	LED ( $E_{\lambda} < 4 \times 10^{-5} \text{ W} \cdot \text{m}^{-2} \cdot \text{nm}^{-1}$ , $L_{\lambda} < 0.05 \text{ W} \cdot \text{m}^{-2} \cdot \text{sr}^{-1} \cdot \text{nm}^{-1}$ )
LED lifetime	10,000 hours
Power	5 Watts
Light intensity	approx. 10.000 lux (in a distance of 3.9 in (10 cm))
Radiation	Output of radiation beyond the visible spectrum (infrared and UV) is so low that it is harmless to eyes and skin.
Operating conditions	32 °F to 95 °F (0 °C to 35 °C), 15–90 % relative humidity
Storage conditions	-4 °F to 122 °F (-20 °C to 50 °C), 15–90 % relative humidity
Product classification	External power supply, Protection class II, IP21
Color temperature	6.500 °K
Adapter Model	UES06WU-120046SPA (DONGGUAN SHILONG FUHUA ELECTRONIC CO LTD)
Power Adapter Input	100–240 V ~50/60 Hz, 0.2 A

Power Adapter Output	12 V DC, 0.46 A
Polarity	 Center Positive

## 10. FCC Compliance Statement

Changes or modifications to the product not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

**NOTE:** This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

## 11. Warranty

### Limited Lifetime Warranty For Original Purchaser

Your Beurer daylight lamp, Model TL 30, excluding the battery, is warranted to be free from defects in materials and workmanship for the life of the product under normal conditions of intended use and service. This warranty extends only to the original retail purchaser and does not extend to retailers or subsequent owners.

We will, at our option, repair or replace the Beurer daylight lamp, Model TL 30, without additional charge, for any part or parts covered by these written warranties. No refunds will be given. Repair or replacement is our only responsibility and your only remedy under this written warranty. If replacement parts for defective materials are not available, Beurer reserves the right to make product substitutions in lieu of repair or replacement.

For warranty service contact our customer service department at **1-800-536-0366** or at **info@beurer.com** to provide a description of the problem. If the problem is deemed to be within the scope of the limited lifetime warranty, you will be asked to mail the product at your costs in its original package with proof of purchase, your name, address and phone number. If the problem is not deemed to be within the scope of the limited lifetime warranty, we will provide a quotation for repair respectively replacement and return shipping fee.

This warranty does not cover damage caused by misuse or abuse; accident; the attachment of unauthorized accessory; alteration to the product; improper installation; misapplication; lack of reasonable care with respect to the product; unauthorized repairs or modifications; improper use of electrical/power supply; old worn batteries; normal wear; loss of power; dropped product; malfunction or damage of an operating part as a result of failure to comply with instructions for use or to provide manufacturer's recommended maintenance; transit damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Beurer. This warranty is void if the product is ever used in a commercial or business environment. The maximum liability of Beurer under this warranty is limited to the purchase price actually paid by the customer for the product covered by the warranty, as confirmed by proof of purchase, regardless of the amount of any other direct or indirect damage suffered by the customer.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adaptation to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

**THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. ANY IMPLIED WARRANTIES, OBLIGATIONS, OR LIABILITIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTY OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED IN DURA-**

**TION TO THE DURATION OF THIS APPLICABLE WRITTEN WARRANTY.** Some states do not allow limitations on how long an implied warranty lasts, so the above limitations may not apply to you.

**IN NO EVENT SHALL BEURER BE LIABLE FOR ANY SPECIAL, INCIDENTAL, INDIRECT OR CONSEQUENTIAL DAMAGES FOR BREACH OF THIS OR ANY OTHER WARRANTY, EXPRESS, IMPLIED OR ANY OTHER THEORY OF LIABILITY, WHATSOEVER.** Some states do not allow the exclusion or limitation of special, incidental, or consequential damages, so the above limitation may not apply to you.

Beurer does not authorize anyone, including, but not limited to, retailers, the subsequent consumer purchaser of the product from a retailer or remote purchaser, to obligate Beurer in any way beyond the terms set forth herein.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate in connection with any products or parts thereof which are repaired, replaced, altered, or modified, without the prior explicitly written consent of Beurer.

This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state.

For more information regarding our product line in the USA, please visit: [www.beurer.com](http://www.beurer.com)

Distributed by:  
Beurer North America LP  
900 N Federal Highway, Suite 300  
Hallandale Beach, FL 33009  
[www.beurer.com](http://www.beurer.com)

**Questions or comments? Call toll free 1-800-536-0366 or contact [info@beurer.com](mailto:info@beurer.com)**

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